

**AIDA**  
autoimmune diagnostic assays

**REF 10514**

**DGP-G**



# Instruction Manual

## Table of Contents

---

1	Intended Use .....	1
2	Clinical Application and Principle of the Assay .....	1
3	Kit Contents .....	2
4	Storage and Shelf Life .....	2
5	Precautions of Use.....	3
6	Sample Collection, Handling and Storage.....	4
7	Assay Procedure .....	4
8	Quantitative Interpretation.....	7
9	Technical Data.....	8
10	Performance Data.....	8
11	Literature .....	9



AIDA GmbH & Co. KG  
Dr.-Karl-Aschoff-Straße 9  
D-55543 Bad Kreuznach  
Tel: +49-671-920 650 90  
Fax: +49-671-920 650 91  
info@aida-diagnostics.com  
www.aida-diagnostics.com

	Product Ref.	10514
	Product Desc.	DGP-G
	Manual Rev. No.	001 : 2014-01-09

## 1 Intended Use

**DGP-G** is a solid phase enzyme immunoassay employing synthetic, deamidated gliadin-derived peptides for the quantitative detection of IgG antibodies against deamidated Gliadin-specific peptides (DGP) in human serum.

The assay is a tool in the diagnosis of celiac disease (gluten-sensitive enteropathy).

## 2 Clinical Application and Principle of the Assay

Gluten-sensitive enteropathy or celiac disease is characterized by atrophy of the small intestinal villi leading to a so-called flat mucosa. It is caused by a pathological intolerance to Gliadin, the alcohol-soluble fraction of gluten in wheat, rye and barley. As celiac disease is caused by the uptake of gluten, consequently a gluten-free diet cures the disease completely and thus has to be maintained for life-time. Renewed consumption of Gliadin leads to a return of the symptoms. The disease is HLA-associated (>95% of patients have DQ2 enREFd by DQA1\*0501 and DQB1\*0201) and manifests at any age with a peak onset in early childhood, even in neonatals. The incidence rates range from 1 in 4000 to 1 in 300 in european countries.

Diagnosis of celiac disease is made by small intestinal biopsy (demonstrating the flat mucosa) supported by serological markers. Antibodies against Gliadin and tissue Transglutaminase (tTG) are of major significance. tTG has been identified as the major target antigen of EMA, antibodies binding to endomysium (extracellular constituent of smooth muscle) in indirect immunofluorescence test (IFT), which has been so far an important tool for the diagnosis of celiac diseases.

Circulating IgG and IgA antibodies to Gliadin are found in the serum of most but not all celiac disease patients, though the specificity of these antibodies are significantly lower compared to tTG and EMA.

Recent work has revealed that gliadin reactive antibodies from celiac patients bind a very limited number of specific epitopes on the gliadin molecule. The selective deamidation of gliadin by tissue transglutaminase results in enhanced binding by anti-gliadin antibodies. Assays using deamidated and defined peptides have been shown to have higher diagnostic accuracy for celiac disease when compared to standard anti-gliadin assays.

The determination of IgG antibodies to Gliadin (and/or tTG) is especially of high value as approximately 2% - 5% of celiac patients display an IgA deficiency, thus being missed by IgA subclass tests.

Moreover, antibodies to Gliadin and DGP may be the only serological marker in neonatals, as anti-tTG and EMA autoantibodies are not present at this age.

### Principle of the test

Serum samples diluted 1:101 are incubated in the microplates coated with the specific antigen. Patient's antibodies, if present in the specimen, bind to the antigen. The unbound fraction is washed off in the following step. Afterwards anti-human immunoglobulins conjugated to horseradish peroxidase (conjugate) are incubated and react with the antigen-antibody complex of the samples in the microplates. Unbound conjugate is washed off in the following step. Addition of TMB-substrate generates an enzymatic colorimetric (blue) reaction, which is stopped by diluted acid (color changes to yellow). The intensity of color formation from the chromogen is a function of the amount of conjugate bound to the antigen-antibody complex and this is proportional to the initial concentration of the respective antibodies in the patient sample.

### 3 Kit Contents

<b>TO BE RECONSTITUTED</b>				
Item	Quantity	Cap color	Solution color	Description / Contents
Sample Buffer (5x)	1 x 20ml	White	Yellow	5 x concentrated Tris, sodium chloride (NaCl), bovine serum albumin (BSA), sodium azide < 0.1% (preservative)
Wash Buffer (50x)	1 x 20ml	White	Green	50 x concentrated Tris, NaCl, Tween 20, sodium azide < 0.1% (preservative)
<b>READY TO USE</b>				
Item	Quantity	Cap color	Solution color	Description / Contents
Negative Control	1 x 1.5ml	Green	Colorless	Human serum (diluted), bovine serum albumin (BSA), sodium azide < 0.1% (preservative)
Positive Control	1 x 1.5ml	Red	Yellow	Human serum (diluted), bovine serum albumin (BSA), sodium azide < 0.1% (preservative)
Calibrators	6 x 1.5ml	White	Yellow *	Concentration of each calibrator: 0, 3, 10, 30, 100, 300 U/ml. Human serum (diluted), bovine serum albumin (BSA), sodium azide < 0.1% (preservative)
Conjugate, IgG	1 x 15ml	Blue	Blue	Containing: Anti-human immunoglobulins conjugated to horseradish peroxidase, bovine serum albumin (BSA)
TMB Substrate	1 x 15ml	Black	Colorless	Stabilized tetramethylbenzidine and hydrogen peroxide (TMB/H <sub>2</sub> O <sub>2</sub> )
Stop Solution	1 x 15ml	White	Colorless	1M Hydrochloric Acid
Microtiter plate	12 x 8 well strips	N/A	N/A	With breakaway microwells. Refer to paragraph 1 for coating.
* Color increasing with concentration				
<b>MATERIALS REQUIRED, BUT NOT PROVIDED</b>				
Microtiter plate reader 450 nm reading filter and recommended 620 nm reference filter (600-690 nm). Glass ware (cylinder 100-1000ml), test tubes for dilutions. Vortex mixer, precision pipettes (10, 100, 200, 500, 1000 µl) or adjustable multipipette (100-1000µl). Microplate washing device (300 µl repeating or multichannel pipette or automated system), adsorbent paper. Our tests are designed to be used with purified water according to the definition of the United States Pharmacopeia (USP 26 - NF 21) and the European Pharmacopeia (Eur.Ph. 4th ed.).				

### 4 Storage and Shelf Life

Store all reagents and the microplate at 2-8°C/35-46°F, in their original containers. Once prepared, reconstituted solutions are stable at 2-8°C/35-46°F for at least 1 month. Reagents and the microplate shall be used within the expiry date indicated on each component, only. Avoid intense exposure of TMB solution to light. Store microplates in designated foil, including the desiccant, and seal tightly.

 autoimmune diagnostic assays	Product Ref.	10514
	Product Desc.	DGP-G
	Manual Rev. No.	001 : 2014-01-09

## 5 Precautions of Use

### 5.1 Health hazard data

**THIS PRODUCT IS FOR IN VITRO DIAGNOSTIC USE ONLY.** Thus, only staff trained and specially advised in methods of in vitro diagnostics may perform the kit. Although this product is not considered particularly toxic or dangerous in conditions of the intended use, refer to the following for maximum safety:

#### ***Recommendations and precautions***

This kit contains potentially hazardous components. Though kit reagents are not classified being irritant to eyes and skin we recommend to avoid contact with eyes and skin and wear disposable gloves.

**WARNING !** Calibrators, Controls and Buffers contain sodium azide ( $\text{NaN}_3$ ) as a preservative.  $\text{NaN}_3$  may be toxic if ingested or adsorbed by skin or eyes.  $\text{NaN}_3$  may react with lead and copper plumbing to form highly explosive metal azides. On disposal, flush with a large volume of water to prevent azide build-up. Please refer to decontamination procedures as outlined by CDC or other local/national guidelines.

**Do not smoke, eat or drink when manipulating the kit. Do not pipette by mouth.**

All human source material used for some reagents of this kit (controls, standards e.g.) has been tested by approved methods and found negative for HbsAg, Hepatitis C and HIV 1. However, no test can guarantee the absence of viral agents in such material completely. Thus handle kit controls, standards and patient samples as if capable of transmitting infectious diseases and according to national requirements.

The kit contains material of animal origin as stated in the table of contents, handle according to national requirements.

### 5.2 General directions for use

In case that the product information, including the labeling, is defective or incorrect please contact the manufacturer or the supplier of the test kit.

Do not mix or substitute reagents or microplates from different lot numbers. This may lead to variations in the results.

Allow all components to reach room temperature (20-32°C/68-89.6°F) before use, mix well and follow the recommended incubation scheme for an optimum performance of the test.

**Incubation: We recommend test performance at 30°C/86°F for automated systems.**

Never expose components to higher temperature than 37°C/ 98.6°F.

Always pipette substrate solution with brand new tips only. Protect this reagent from light. Never pipette conjugate with tips used with other reagents prior.

**A definite clinical diagnosis should not be based on the results of the performed test only, but should be made by the physician after all clinical and laboratory findings have been evaluated. The diagnosis is to be verified using different diagnostic methods.**

 autoimmune diagnostic assays	Product Ref.	10514
	Product Desc.	DGP-G
	Manual Rev. No.	001 : 2014-01-09

## 6 Sample Collection, Handling and Storage

Use preferentially freshly collected serum samples. Blood withdrawal must follow national requirements. Do not use icteric, lipemic, hemolysed or bacterially contaminated samples. Sera with particles should be cleared by low speed centrifugation (<1000 x g). Blood samples should be collected in clean, dry and empty tubes.

After separation, the serum samples should be used during the first 8h, respectively stored tightly closed at 2-8°C/35-46°F up to 48h, or frozen at -20°C/-4°F for longer periods

## 7 Assay Procedure

### 7.1 Preparations prior to starting

Dilute concentrated reagents:

Dilute the concentrated sample buffer 1:5 with distilled water (e.g. 20 ml plus 80 ml).

Dilute the concentrated wash buffer 1:50 with distilled water (e.g. 20 ml plus 980 ml).

To avoid mistakes we suggest to mark the cap of the different calibrators.

#### **Samples:**

Dilute serum samples 1:101 with sample buffer (1x)

e.g. 1000 µl sample buffer (1x) + 10 µl serum. Mix well !

#### **Washing:**

Prepare 20 ml of diluted wash buffer (1x) per 8 wells or 200 ml for 96 wells

e.g. 4 ml concentrate plus 196 ml distilled water.

#### **Automated washing:**

Consider excess volumes required for setting up the instrument and dead volume of robot pipette.

#### **Manual washing:**

Discard liquid from wells by inverting the plate. Knock the microwell frame with wells downside vigorously on clean adsorbent paper. Pipette 300 µl of diluted wash buffer into each well, wait for 20 seconds. Repeat the whole procedure twice again.

#### **Microplates:**

Calculate the number of wells required for the test. Remove unused wells from the frame, replace and store in the provided plastic bag, together with desiccant, seal tightly (2-8°C/35-46°F).

## 7.2 Pipetting Scheme

We suggest pipetting calibrators, controls and samples as follows:

For *QUANTITATIVE* interpretation

	1	2	3	4...
A	Cal A	Cal E	P1	
B	Cal A	Cal E	P1	
C	Cal B	Cal F	P2	
D	Cal B	Cal F	P2	
E	Cal C	PC	P3	
F	Cal C	PC	P3	
G	Cal D	NC	...	
H	Cal D	NC	...	

CalA: calibrator A

CalB: calibrator B

CalC: calibrator C

CalD: calibrator D

CalE: calibrator E

CalF: calibrator F

PC: positive control



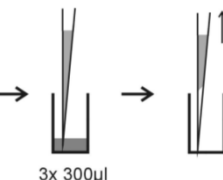
NC: negative control


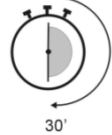
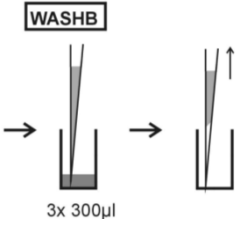

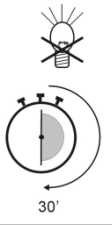
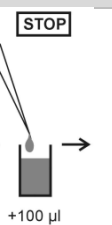

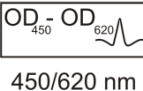
P1: patient 1

P2: patient 2

P3: patient 3

## 7.3 Test Steps

Step	Description
1.	Ensure preparations from step 7.1 above have been carried out prior to pipetting.
2.	Use the following steps in accordance with quantitative interpretation results desired:
<b>CONTROLS &amp; SAMPLES</b>	
3.	<div style="display: flex; align-items: center;">  <div style="margin-left: 20px;"> <p>Pipette into the designated wells as described in chapter 7.2 above, 100 µl of either:</p> <p style="margin-left: 20px;">Calibrators (CAL.A to CAL.F)</p> <p>and 100 µl of each of the following:</p> <ul style="list-style-type: none"> <li>Negative control (NC) and Positive control (PC), and</li> <li>Patients diluted serum (P1, P2...)</li> </ul> </div> </div>
4.	<div style="display: flex; align-items: center;">  <div style="margin-left: 20px;"> <p>Incubate for 30 minutes at 20-32°C/68-89.6°F.</p> </div> </div>
5.	<div style="display: flex; align-items: center;"> <div style="margin-right: 20px;"> <p><b>WASHB</b></p>  </div> <div> <p>Wash 3x with 300 µl washing buffer (diluted 1:50).</p> </div> </div>

CONJUGATE	
6.	<div style="display: flex; align-items: center;"> <div style="margin-right: 20px;">  </div> <div> <p>Pipette 100 µl conjugate into each well.</p> </div> </div>
7.	<div style="display: flex; align-items: center;"> <div style="margin-right: 20px;">  </div> <div> <p>Incubate for 30 minutes at 20-32°C/68-89.6°F.</p> </div> </div>
8.	<div style="display: flex; align-items: center;"> <div style="margin-right: 20px;">  </div> <div> <p>Wash 3x with 300 µl washing buffer (diluted 1:50).</p> </div> </div>
SUBSTRATE	
9.	<div style="display: flex; align-items: center;"> <div style="margin-right: 20px;">  </div> <div> <p>Pipette 100 µl TMB substrate into each well.</p> </div> </div>
10.	<div style="display: flex; align-items: center;"> <div style="margin-right: 20px;">  </div> <div> <p>Incubate for 30 minutes at 20-32°C/68-89.6°F, protected from intense light.</p> </div> </div>
STOP	
11.	<div style="display: flex; align-items: center;"> <div style="margin-right: 20px;">  </div> <div> <p>Pipette 100 µl stop solution into each well, using the same order as pipetting the substrate.</p> </div> </div>
12.	<div style="display: flex; align-items: center;"> <div style="margin-right: 20px;">  </div> <div> <p>Incubate 5 minutes minimum.</p> </div> </div>
13.	<p>Agitate plate carefully for 5 sec.</p>
14.	<div style="display: flex; align-items: center;"> <div style="margin-right: 20px;">  </div> <div> <p>Read absorbance at 450 nm (recommended 450/620 nm) within 30 minutes.</p> </div> </div>



	Product Ref.	10514
	Product Desc.	DGP-G
	Manual Rev. No.	001 : 2014-01-09

## 8 Quantitative Interpretation

For **quantitative interpretation** establish the standard curve by plotting the **optical density (OD) of each calibrator (y-axis)** with respect to the corresponding concentration values in U/ml (x-axis). For best results we recommend log/lin coordinates and 4-Parameter Fit. From the OD of each sample, read the corresponding antibody concentrations expressed in U/ml.

Normal Range	Equivocal Range	Positive Results
< 12 U/ml	12 - 18 U/ml	>18 U/ml

### *Example of a standard curve*

**Do NOT use this example for interpreting patient's result**

Calibrators IgG	OD 450/620 nm	CV % (Variation)
0 U/ml	0.039	0.0
3 U/ml	0.135	1.1
10 U/ml	0.313	3.2
30 U/ml	0.552	3.6
100 U/ml	1.187	3.5
300 U/ml	2.031	3.2

### *Example of calculation*

Patient	Replicate (OD)	Mean (OD)	Result (U/ml)
P 01	1.017/1.029	1.023	85.6
P 02	0.498/0.494	0.496	29.4

Samples above the highest calibrator range should be reported as >Max. They should be diluted as appropriate and re-assayed. Samples below calibrator range should be reported as < Min.

For lot specific data, see enclosed quality control leaflet. Medical laboratories might perform an in-house quality control by using own controls and/or internal pooled sera, as foreseen by national regulations.

Each laboratory should establish its own normal range based upon its own techniques, controls, equipment and patient population according to their own established procedures.

In case that the values of the controls do not meet the criteria the test is invalid and has to be repeated.

The following technical issues should be verified: Expiration dates of (prepared) reagents, storage conditions, pipettes, devices, photometer, incubation conditions and washing methods.

If the items tested show aberrant values or any kind of deviation or that the validation criteria are not met without explicable cause please contact the manufacturer or the supplier of the test kit.

	Product Ref.	10514
	Product Desc.	DGP-G
	Manual Rev. No.	001 : 2014-01-09

## 9 Technical Data

Sample material:	serum
Sample volume:	10 µl of sample diluted 1:101 with 1x sample buffer
Total incubation time:	90 minutes at 20-32°C/68-89.6°F
Calibration range:	0-300 U/ml
Analytical sensitivity:	1.0 U/ml
Storage:	at 2-8°C/35-46°F use original vials only.
Number of determinations:	96 tests

## 10 Performance Data

### 10.1 Analytical sensitivity

Testing sample buffer 30 times on *DGP-G* gave an analytical sensitivity of 1.0 U/ml.

### 10.2 Specificity and sensitivity

The microplates are coated with synthetic, deamidated gliadin-derived peptides.

### 10.3 Linearity

Chosen sera have been tested with this kit and found to dilute linearly. However, due to the heterogeneous nature of human autoantibodies there might be samples that do not follow this rule.

Sample No.	Dilution Factor	Measured (U/ml)	Expected (U/ml)	Recovery (%)
1	1 / 100	66.1	64.8	102.1
	1 / 200	33.6	32.4	103.7
	1 / 400	14.1	16.2	87.0
	1 / 800	7.3	8.1	90.1
2	1 / 100	25.3	23.2	109.1
	1 / 200	11.7	11.6	100.9
	1 / 400	4.8	5.8	82.8
	1 / 800	2.5	2.9	86.2

	Product Ref.	10514
	Product Desc.	DGP-G
	Manual Rev. No.	001 : 2014-01-09

## 10.4 Precision

To determine the precision of the assay, the variability (intra and inter-assay) was assessed by examining its reproducibility on three serum samples selected to represent a range over the standard curve.

Intra-assay		
Sample No.	Mean (U/ml)	CV (%)
1	24.8	2.6
2	71.5	3.9
3	140.6	3.6

Inter-assay		
Sample No.	Mean (U/ml)	CV (%)
1	28.2	1.1
2	82.8	5.1
3	168.7	3.9

## 10.5 Calibration

Due to the lack of international reference calibration this assay is calibrated in arbitrary units (U/ml).

## 11 Literature








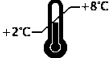












**Schwartz E**, et al.: Serologic assay based on gliadin-related nonapeptides as a highly sensitive and specific diagnostic aid in celiac disease. *Clinical Chemistry* 50: 2370-2375, 2004.

**Osman AA**, et al.: B-Cell epitopes of gliadin. *Clin Exp Immunol* 121: 248-254, 2000.

**Mothes T.**: Deamidated gliadin peptides as targets for celiac disease-specific antibodies. *Adv Clin Chem.* 2007; 44: 35-63.

**Aleanzi M, et al.**: Celiac disease: Antibody recognition against native and selectively deamidation gliadin peptides. *Clinical Chemistry* 47: 2023-2028, 2001.



	- Diagnosi in vitro - Pour diagnostic in vitro - In Vitro Diagnostikum - Para uso Diagnóstico in vitro	- For in vitro diagnostic use - Para uso diagnóstico in vitro - In Vitro Διαγνωστικό μέσο
	° Numero d'ordine ° Référence Catalogue ° Bestellnummer ° Número de catálogo	° Catalogue number ° Numéro de catálogo ° Αριθμός παραγγελίας
	° Descrizione lotto ° Lot ° Chargen Bezeichnung ° Lote	° Lot ° Lote ° Χαρακτηρισμός παρτίδας
	° Conformità europea ° Déclaration CE de Conformité ° Europäische Konformität ° Declaração CE de Conformidade	° EC Declaration of Conformity ° Declaración CE de Conformidad ° Ευρωπαϊκή συμφωνία
	° 96 determinazioni ° 96 tests ° 96 Bestimmungen ° 96 Testes	° 96 tests ° 96 pruebas ° 96 προσδιορισμοί
	° Rispettare le istruzioni per l'uso ° Voir les instructions d'utilisation ° Gebrauchsanweisung beachten ° Ver as instruções de uso	° See instructions for use ° Ver las instrucciones de uso ° Λάβετε υπόψη τις οδηγίες χρήσης
	° Da utilizzarsi entro ° Utilise avant le ° Verwendbar bis ° Utilizar antes de	° Use by ° Utilizar antes de ° Χρήση μέχρι
	° Conservare a 2-8°C ° Conserver à 2-8°C ° Lagerung bei 2-8°C ° Conservar entre 2-8°C	° Store at 2-8°C (35-46°F) ° Conservar a 2-8°C ° Φυλάσσεται στους 2-8°C
	° Prodotto da ° Fabriqué par ° Hergestellt von ° Fabricado por	° Manufactured by ° Fabricado por ° Κατασκευάζεται από
	° Calibratore cut-off ° Etalon Seuil ° Grenzwert Kalibrator ° Calibrador de cut-off	° Cut off Calibrator ° Calibrador de cut-off ° Οριακός ορός Αντιδραστήριο βαθμονόμησης
	° Controllo positivo ° Contrôle Positif ° Positiv Kontrolle ° Controlo positivo	° Positive Control ° Control Positivo ° Θετικός ορός ελέγχου
	° Controllo negativo ° Contrôle Négatif ° Negativ Kontrolle ° Controlo negativo	° Negative Control ° Control Negativo ° Αρνητικός ορός ελέγχου
	° Calibratore ° Etalon ° Kalibrator ° Calibrador	° Calibrator ° Calibrador ° Αντιδραστήριο βαθμονόμησης
	° Recupero ° Corrélation ° Wiederfindung ° Recuperação	° Recovery ° Recuperado ° Ανάκτηση
	° Coniugato ° Conjugé ° Konjugat ° Conjugado	° Conjugate ° Conjugado ° Σύζευγμα
	° Micropiastra rivestita ° Microplaque sensibilisée ° Beschichtete Mikrotiterplatte ° Microplaca revestida	° Coated microtiter plate ° Microplaca sensibilizada ° Επικαλυμμένη μικροπλάκα
	° Tampone di lavaggio ° Tampon de Lavage ° Waschpuffer ° Solução de lavagem	° Wash buffer ° Solución de lavado ° Ρυθμιστικό διάλυμα πλύσης
	° Tampone substrato ° Substrat ° Substratpuffer ° Substrato	° Substrate buffer ° Tampón sustrato ° Ρυθμιστικό διάλυμα υποστρώματος
	° Reagente bloccante ° Solution d'Arrêt ° Stopreagenz ° Solução de paragem	° Stop solution ° Solución de parada ° Αντιδραστήριο διακοπής αντίδρασης
	° Tampone campione ° Tampon Echantillons ° Probenpuffer ° Diluente de amostra	° Sample buffer ° Tampón Muestras ° Ρυθμιστικό διάλυμα δειγμάτων