

Detect neutralizing Antibodies (nAbs) against SARS-CoV-2 in just 1 hour

PhoenixDx® SARS-CoV-2 Neutralizing Antibody Test Kit


Neutralizing antibodies (nAbs) are antibodies that block SARS-CoV-2 from infecting the host cell and therefore are an important factor to evaluate the immune status of a patient after vaccination or recovery from COVID-19.

With **PhoenixDx® SARS-CoV-2 Neutralizing Antibody Test Kit** you can test **serum or plasma** for the presence of nAbs with a quick & easy ELISA assay in 1 hour.

PhoenixDx® SARS-CoV-2 Neutralizing Antibody Test

Application


- well-established ELISA Assay System
- works with standard laboratory equipment
- easy to execute
- easy to analyze
- **fast - 96 reactions in 1 h**

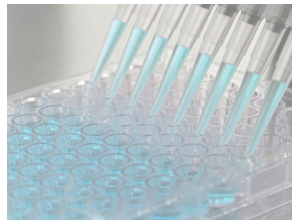

- detects nAbs in the serum of **vaccinated people**
- detects nAbs in the serum of **people recovering from COVID-19**
- for use with **serum or plasma** samples




While regular ELISA Assays detect the entirety of anti-SARS-CoV-2 antibodies, **PhoenixDx® SARS-CoV-2 Neutralizing Antibody Test Kit** detects very specifically only those that prevent the interaction between the receptor binding domain of SARS-CoV-2 spike protein (**RBD**) and the host cell receptor (**ACE2**).

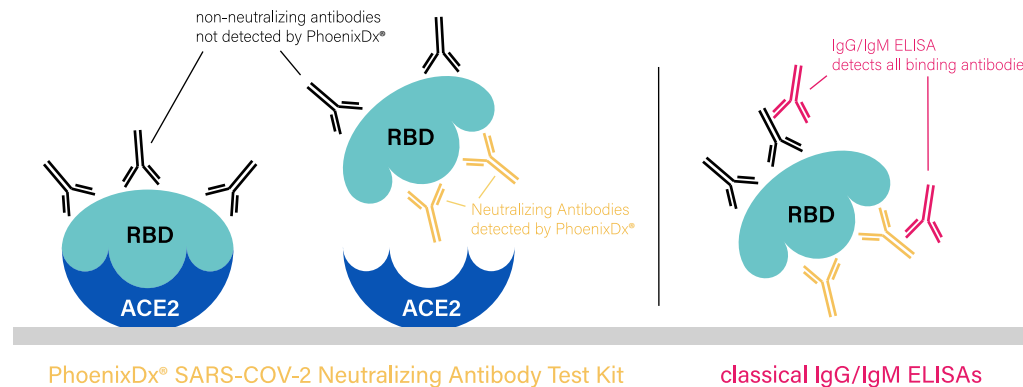


Execution

Use and analysis of **PhoenixDx® SARS-CoV-2 Neutralizing Antibody Test Kit** is as easy as an regular ELISA assay:

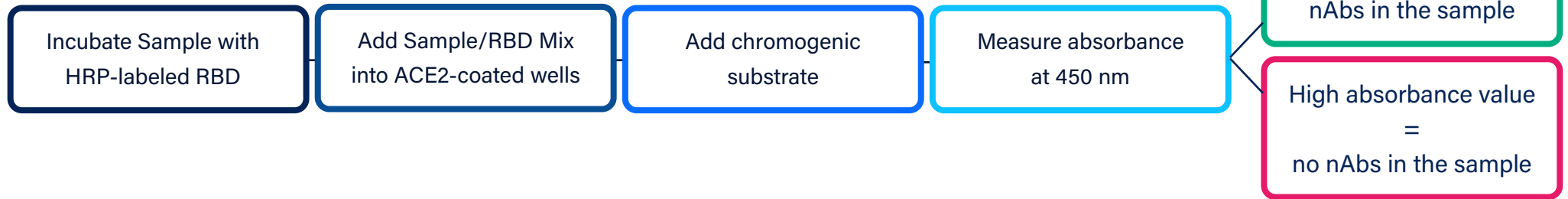

Performance Data

The performance of **PhoenixDx® SARS-CoV-2 Neutralizing Antibody Test Kit** was evaluated using the WHO's Anti-SARS-CoV-2 Verification Panel for Serology Assays, NIBSC Code 20/B770, as at 22/09/2020. All samples of the Verification Panel were tested correctly (23 positive and 14 negative samples). Comparison to a competitor product showed identical detection of positive and negative samples.

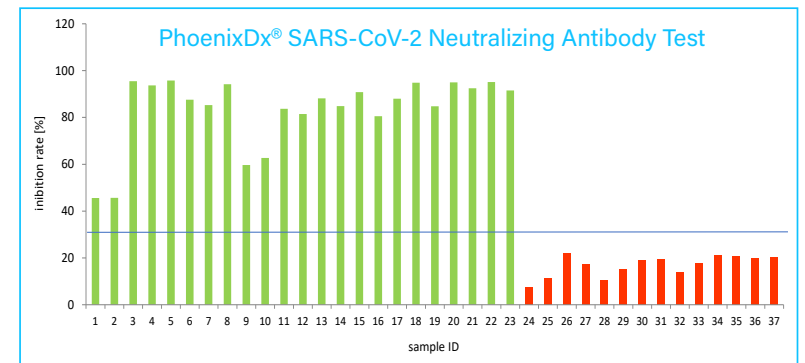

Contact and Ordering

	Tests	Catalogue Number
PhoenixDx® SARS-CoV-2 Neutralizing Antibody Test Kit (ELISA)	96 Tests	PCCSKU18005

For ordering **PhoenixDx® SARS-CoV-2 Neutralizing Antibody Test Kit** and related products, contact us at

sales@procomcure.com

