de	Gebrauchsanweisung	2
en	User's Manual	5
fr	Mode d'emploi	8
it	Istruzioni per l'uso	11
es	Instrucciones de empleo	14
pt	Manual de operação	17
ru	Руководство по применению	20

Basic-Set

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User's Manual Basic Diagnostic Set

Dear Customers! We thank you for choosing a KaWe-product. Our products are known for their high quality and long life. This KaWe product meets EC Standards 93/42/EWG (Standards for Medical Products).

This is a class 1 product according to Standards 93/42/EWG and is intended for repeated use on different patients. Any other use is not in accordance with the intended use of the unit. The manufacturer is not liable for any resultant damages. The manufacturer is also not liable if the unit is not checked before its use.

Please read these instructions thoroughly and carefully before attempting to use this product and heed the given instructions.

Familiarise yourself fully with this product before using it.

Utilization: The Basic Diagnostic Sets are only to be used by authorized personnel.

Function: The Basic Diagnostic Set is to be used for the direct otorhinolaryngological inspection of the larynx, the auditory canal and the nose. It is also used for direct ophthalmoscopic inspection of the eyes and/or the fundus of the eyes.

Risks/Warning of possible dangers: 1. Risk of injury of the sensitive mucosa. 2. After use, please clean the handles with an alcohol-based disinfectant solution in order to avoid the risk of infection.

Replacing the otoscope lamp: Remove the plastic cap from the lamp and remove the lamp from the otoscope head. If necessary, clean the glass bulb of the new lamp with alcohol. The glass bulb must be properly clean and free of finger prints (free of grease). The lamp must be pushed carefully into the provided slot until it touches the lateral pin.

Replacing the ophthalmoscope lamp: Satisfactory operation of the unit can only be guaranteed with the use of original KaWe incandescent lamps that ensure an optimal light quality! Loosen the clamping screw and remove the ophthalmoscope head from the handle (1). Remove the lamp holder (2) from the head. Now replace the bulb. If necessary, clean

the glass bulb (3) of the new lamp with alcohol. The glass bulb should be clean and free of finger prints (grease free). Insert the new lamp with the lamp holder in the head. Now, rescrew the union ring. The KaWe ophthalmoscope is designed in such a way that it can also be used without pupil dilating medication. In this case however, the visual field of the patient is reduced. The illumination brightness of the KaWe EUROLIGHT® E10 can be adjusted. A reduction of the brightness causes an enlargement of the pupil and the patient will be less blinded by the light. Ametropia can be corrected with the device's correction wheel by either the patient or the doctor.

Use (Ophthalmoscope + Otoscope): Satisfactory operation is guaranteed when standard 2.5V KaWe handles are used with 2 batteries ("C" Baby 1.5V. We recommend the use of alkaline batteries) or a KaWe-NiMH 2.5V battery. Slip the instrument head onto the unit and lock it into place. Turn on the sliding switch by pushing it to the left. The light intensity can be easily adjusted with the rheostat (maximum switch position). Turn off the sliding switch by returning it to the right side and back to its starting position. Replace the battery/accumulator by opening the bottom-locking cap. Should you need further information, please contact your dealer!

Cleaning: The KaWe otoscope and the ophthalmoscope can be easily cleaned with water, alcohol or soap. Please use the otoscope and the ophthalmoscope only when they are well cleaned in order to avoid possible risk of infection. In order to avoid contact with dirt and dust, please always store the otoscope and ophthalmoscope in their packaging. They can be cleaned externally with a damp, soft and lint-free cloth.

Disinfection: Disinfection is the first treatment that is to be carried out on contaminated objects and materials with the goal of reducing the microorganisms and further easing the cleaning process. It is extremely important to prevent the impurities from drying onto the instrument. Disinfection also has the goal of protecting the personnel during the treatment of the instruments as well as the surrounding environment. The instruments can be cleaned and moistened with an alcohol-based disinfecting agent. Heed the instructions given by the manufacturer of the disinfecting agent such as reaction time etc. Never allow the otoscope and the ophthalmoscope to come into contact with liquid and make sure that no liquid penetrates the housing! Due to the fact that the otoscope and the ophthalmoscope are not intended for use in surgical procedures, simple cleaning and/or disinfection with an alcohol-based surface disinfectant is sufficient. Do not use any abrasive cleansing agents! The otoscope and ophthalmoscope can not be sterilized!

Sterilization: Multi-use ear funnel; laryngeal mirror, nasal speculum, lamp holder (without lamp) and plastic blade holder can be sterilized in an autoclave at 134°C. Before sterilizing, remove larger-sized impurities

by rinsing with running water or in a soap bath. Dry objects before placing them into the autoclave. Be sure not to exceed the allowed capacity of the autoclave.

Guarantee: When used under normal circumstances and with attention to these instructions we guarantee this product for two years after the date of purchase. Should you need further information or should your instrument require repair, please contact your dealer

Technical Data:

Ophthalmoscope	Otoscope
Ametropia correction: ± 20 dpt..	3-way extendable magnifying glass
Angle of sight: ± 6°	High-power vacuum bulbs: 2.5V / 0.81W
Aspherical lens for optimal focusing quality	
Aspherical condenser for optimal illumination	
Light strength of vacuum bulb: 2.5V / 0.81W	

User's Manual/Maintenance Instructions for Rechargeable NiMH/Li-Ion Batteries

Purpose: The charging batteries are to be used as an energy source for the relevant KaWe products.

Unsuited use/contraindication of the products: Any use other than that described here is not in accordance with the intended use of this product. The manufacturer is not liable for any resulting damages. The user alone bears the risk.

Risks and notice of possible danger: **1.** Avoid short-circuiting the rechargeable batteries. **2.** Never allow the rechargeable batteries to come into contact with liquids.

Further information, maintenance, storage: The rechargeable batteries can be cleaned with a damp cloth and then be wiped off with a dry cloth. Please ensure that no short-circuit is produced.

Further maintenance instructions that may prolong the lifetime of the rechargeable battery: Please heed the following instructions: The non-observance of these instructions may result in the battery losing its full capacity within a short period of time. With NiMH technology, a lazy battery effect can occur that is analogous to the classic memory effect. It is triggered by the energy that has not been completely spent prior to recharging the battery. This effect can be nearly eliminated by completely discharging the battery from time to time and then by re-

charging it two or three times in quick succession. Subsequently, the accumulator will regain its full capacity. If the accumulator no longer provides the amount of energy necessary for powering the unit, immediately turn off the unit and charge the battery (in order to prevent it from losing its charge completely, which would result in the destruction of the battery).

Please keep in mind that the rechargeable batteries can spontaneously discharge. When the batteries are not used (charged) for a long period of time, a self discharge may occur. In this case, the capacity will be lost and the batteries may become permanently defective. The manufacturer is not liable for any resulting damages. Therefore, when stored for more than one month, the batteries should be partially charged to approx. 50%.

Please always respect the given charging time in order to prevent overcharging and the resulting destruction of the battery (the KaWe charging station KaWe Med-Charge® 3000 + 4000 has an integrated automatic shut-off). NiMH rechargeable batteries lose their capacity at low temperatures (as low as 0°C).

1. Keep the rechargeable batteries away from heating installations and open flame. **2.** Batteries are not to be immersed in water or salt water. **3.** Never try to disassemble or modify the rechargeable batteries or expose them to heat. **4.** Avoid dropping the batteries and protect them against being hit or crushed as this may damage the housing. **5.** Only use the rechargeable batteries and accessory parts recommended by KaWe. **6.** When using a rechargeable battery for the first time, please make sure to charge it completely.

The use of rechargeable batteries that are not explicitly approved for these units may cause explosions or leakages and the risk of fire, injuries and damage to surroundings. In the case that a rechargeable battery begins to leak and spilled substances come into contact with eyes, mouth, skin or clothing, immediately rinse the affected area with water and consult a doctor.

1. When charging is completed and the charging unit and the compact adapter are no longer in use, please disconnect them from the unit and the socket in order to avoid the risk of fire and other dangers. **2.** Do not cover the charging unit with any kind of material (tablecloths, carpets, bedclothes etc.) while operating the instrument. If you use the adapter for a long period of time, it may become overheated, lose its original shape or catch fire. **3.** Please only charge the rechargeable batteries in the appropriate battery charging unit. The charging unit and the compact adapter have been designed especially for use with your unit. Please do not use them with other products or rechargeable batteries. All other products may cause the risk of excessive heat and distortion and thus the risk of fire and electric shock. **4.** Prior to disposing of the rechargeable battery, ensure that the contacts are masked with adhesive strips or other insulating material in order to prevent their direct contact with other materials. Contact with other metallic material in rubbish bins may cause a fire and injury.

Charging time:

REF	12.80110.712	12.80110.722	
Capacity [mAh]	1600	3000	
Charging time [h]	approx. 8*	approx. 4*	
	12.80120.712	12.80220.722	12.80120.742
	750	2200	700
	approx. 4*	approx. 10*	approx. 12**

*Charging time in the KaWe charging station KaWe MedCharge® 3000 (does not apply to Li-Ion) or 4000 after complete discharge of the battery.

**Charging time in a wall socket after complete discharge of the battery.

Disposal: Please dispose of your rechargeable batteries at a separate location for electric and electronic devices.

Disposal: Please dispose of expired NiMH rechargeable batteries and other batteries in accordance with legal regulations.

Common accessories: Further information about these products can be obtained in our Internet shop at: www.kawemed.de.

Manufacturer: **KaWe**

Contacting the manufacturer: Address or telephone number of the dealer or dial +49-7141-68188-0.

Symbol key:

	Manufacturer
	Date of manufacture
	Heed the User's Manual
	Caution!
	Separate disposal of electric and electronic devices
	Battery disposal
	Complies with relevant EU guidelines

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KIRCHNER & WILHELM GmbH + Co. KG