de	Gebrauchsanweisung – Diagnostikleuchten	2
en	User's Manual – Diagnostic Lights	4
fr	Mode d'emploi – Lampes de diagnostic	6
it	Istruzioni per l'uso – Lampadine diagnostiche	8
es	Instrucciones de empleo – Lámparas de diagnóstico	10
pt	Manual de operação – Lâmpadas de diagnóstico	12
ru	Руководство по применению – фонарики диагностические ...	14

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Cliplight / Cliplight LED / KaWe DIALIGHT® XL / Diagnostikleuchte, weiß

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User's Manual: Cliplight / Cliplight LED / KaWe DIALIGHT® XL / Diagnostic light, white

Dear Customer, we thank you for choosing a KaWe product. Our products are known for their high quality and longevity. This KaWe product meets EC standards 93/42/EEC (standards for medical products).

 Please read this User's Manual thoroughly and carefully before attempting to use this product and heed the given care instructions. Familiarize yourself thoroughly with the operation of this unit before using it.

Retain this User's Manual for future use and pass it on to the next user of this device.

Use:

This diagnostic light is only to be used by authorized professionals.

Intended use:

These lights are used during diagnostic procedures to illuminate areas of the body to be examined such as, for example, inside the mouth or the surface of the skin.

The Cliplight LED diagnostic light is approved for eye examinations (Photobiologically safety tested in accordance with standard DIN EN 62471:2009-03 for lamps and lamp systems).

Operation:

Cliplight diagnostic light: To turn on, press the clip against the shaft of the pen until contact is made with the metal ring on the shaft. To replace the batteries, unscrew the metal lamp socket.

KaWe DIALIGHT® XL diagnostic light: To turn the light on and off, slide the black plastic switch back and forth. To replace the lamp or batteries, unscrew and remove the head from the handle.

Diagnostic light, white: To turn on and off, press the black button at the top of the pen. To replace the lamp or batteries, unscrew the black clip part of the pen.

Unsuitable use/contraindication of the products:

Any use other than that described here is not in accordance with the intended use. The manufacturer is not liable for any resulting damages.

The Cliplight, DIALIGHT® XL and Diagnostic light - white diagnostic lights may not be used for eye examinations!

Warning:

Never allow the lamps to come into contact with fluids. No direct contact with the tongue or mucus membranes.

When examining the mouth cavity ensure that the light does not fall into the mouth due to the risk of suffocation!

If the lights show visible signs of damage, they are to be removed from clinical use.

The Cliplight, DIALIGHT® XL and Diagnostic light - white diagnostic lights may not be used for eye examinations!

Warranty:

When used properly and in accordance with the User's Manual we warranty this product for two years from the date of purchase (except bulbs/rechargeable batteries). For necessary repairs, please contact your dealer.

Limits of reuse preparation:

No special requirements. It is recommended that the instruments be cleaned as soon as possible after being used.

Storage and transport:

Please store the lights in their packaging in order prevent them from collecting dirt or dust. Caution: The lights may break if dropped.

Cleaning:

The surface of the lights may be cleaned with damp or dry cleaning rags.

Disinfection:

The surface of the lights may be disinfected with a cloth moistened with alcohol.

Sterilization:

Sterilization not possible.

Inspection and functionality test:

Visually inspect the light before using it.

Additional information:

This product is to be disposed of at a separate collection location for electric and electronic devices.

Common accessories:

Further information about these articles can be found on our homepage at: www.kawemed.de

Manufacturer:

Kirchner & Wilhelm GmbH & Co. KG Eberhardstraße 56 | DE-71679 Asperg

Contacting the manufacturer:

Address or telephone number of the dealer or dial +49 7141 68188-0

Symbol key:

	Manufacturer
	Date of manufacture
	Lot code
	Heed the User's Manual
	Caution!
	Separate disposal of electric and electronic devices
	Dispose of expired batteries separately at an appropriate collection centre
	Complies with relevant EU guidelines

**Mode d'emploi :****Cliplight / Cliplight LED / KaWe DIALIGHT® XL / Lampe de diagnostic, blanche**

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KIRCHNER & WILHELM GmbH + Co. KG