- de** Gebrauchsanweisung
- en** User's Manual
- fr** Mode d'emploi
- it** Istruzioni per l'uso
- es** Instrucciones de empleo
- pt** Manual de operação
- ru** Руководство по применению

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KaWe

CE

QM-1-024H

50 cm³/cm³

100 cm³/cm³

150 cm³/cm³

CE

de	Gebrauchsanweisung Ganzmetallspritze	4
en	User's Manual All-metal syringes	8
fr	Mode d'emploi Seringue auriculaire métallique	12
it	Istruzioni per l'uso Schizzettone tutto metallo	16
es	Instrucciones de empleo Jeringa auricular metálica	20
pt	Manual de operação Seringa metálica	24
ru	Руководство по применению Шприц металлический	28

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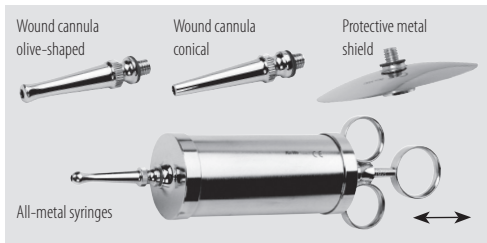
User's Manual All-metal Schimmelbusch syringes (50 - 150 cm³)

Dear Customers, thank you for choosing a KaWe product. Our products are known for their high quality and longevity. This KaWe product fulfils EC Standards 93/42/EEC (standards for medical products).

Please read this User's Manual thoroughly and carefully before attempting to use this product and heed the given care instructions.

Designation: All-metal syringe with wound and enema cannula. The spraying volume (50 cm³, 100 cm³ or 150 cm³) is stated on the package.

Use: The all-metal syringes are only to be used by authorized personnel.

Purpose: Otorhinolaryngologic syringe for rinsing the external and the middle auditory canal.

Unsuited Use/Contraindication of the Products: Any use other than that described

here is not in accordance with the intended use of the instrument. The manufacturer is not liable for any resulting damages. The user alone bears the risk.

Further Information, Maintenance, Storage: When used and stored properly, the all-metal syringes will serve you many years.

Guarantee: When used properly and in accordance with the User's Manual we guarantee this product for two years from the date of purchase. Should you need further information or should your instrument require repair, please contact your dealer.

Operation For the desired application, a protective metal shield and a conical or olive-shaped wound cannula can be screwed on the all-metal syringe. Manufacturer: KaWe

Unit(s): All reusable instruments delivered by KaWe that are fixed constructions (no movable parts) and simple link constructions with exception of all instruments with an aluminium alloy.

Warning: Special attention must be paid to the cleaning of reusable instruments (long insertions of canulas and blind holes). Do not exceed 134°C.

Limitation of reuse preparation: Frequent reprocessing has little effect on these instruments. The end of the product's working life is normally determined by wear and damage caused by use.

Instructions

Workstation: Remove dirt accumulation on surfaces with a disposable cloth.

Storage and transport: No special requirements. It is recommended that the instruments be reprepared immediately after their use.

Cleaning preparation: No special requirements.

Manual cleaning: Unscrew the piston from the syringe and remove it. Clean both parts with running water. Next, apply a medical surface disinfectant and cleaning agent following the concentrations specified by the manufacturer.

Automatic cleaning: Mechanically clean syringes with a neutral or mild alkaline solution. When mechanically cleaning the instrument, only use appropriate processing machines that are suitable for the intended purpose. Remove the piston from the syringe.

Disinfection: The entire syringe and cannula can be disinfected. The concentration and reaction times given by the manufacturer must be followed.

Maintenance: Defective parts must be sorted out and/or replaced. After sterilization, the silicone ring of the piston is to be greased slightly with silicone grease.

Check and performance test: The smooth running of all movable parts as in piston rods and cannulae must be ensured. For all instruments: carry out visual check of the parts with regard to damages and wear.

Packaging: Individual packaging: A standard polyethylene bag can be used. The size of the bag must correspond to the size of the instrument so that it closes properly.

Sterilization: The effectiveness of the used cleaning and disinfecting materials must be pre-approved (for example in the list from the Robert-Koch-Institute/DGHM tested and approved disinfectants and methods) and always validated. Disinfect the syringe in accordance with the following sterilization process:

Recommended process: steam sterilization. Validated in accordance with EN ISO 17665 and/or the DGKH standards for the validation and routine control of sterilization processes with humid heat for medical products. Sterilization temperature 121°C (holding time 20 minutes minimum) or 134°C (holding time 7 min. minimum) 137°C maximum.

Storage: No special requirements.

Additional information: When sterilizing more than one instrument in the autoclave, the maximum capacity specified by the manufacturer of the autoclave may not be exceeded.

Common Accessories: Further information about accessories can be found on our homepage at: www.kawemed.de.

Manufacturer: **KaWe**

Contacting the manufacturer: Address or phone number of the dealer or dial the number +49-7141-68188-0.

Symbol key:

	Manufacturer		Heed the User's Manual
	Date of manufacture		Complies with relevant EU-guidelines
	Caution!		Product reference number
	Lot code		GOST-R certification of exports to Russia

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KIRCHNER & WILHELM GmbH + Co. KG