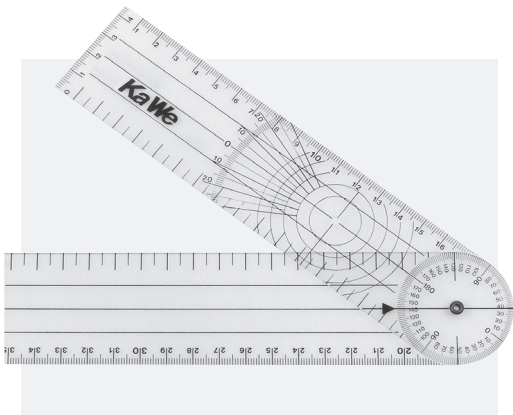
- de** Gebrauchsanweisung
- en** User's Manual
- fr** Mode d'emploi
- it** Istruzioni per l'uso
- es** Instrucciones de empleo
- pt** Manual de operação
- ru** Руководство по применению

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
KaWe

Goniometer

QM-1-161L

de	Goniometer	4
en	Goniometer	6
fr	Goniomètre	9
it	Goniometro	12
es	Goniómetro	15
pt	Goniómetro	18
ru	Гониометр	21

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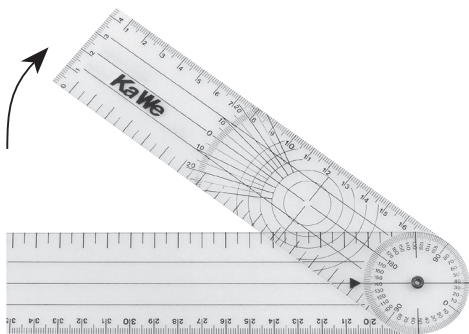Dear Customers, thank you for choosing a KaWe product. Our products are known for their high quality and long life. This is not a medical product.

Please read this User's Manual thoroughly and carefully before attempting to use this product and heed the care instructions.

Use: The goniometers are only to be used by authorized personnel and are exclusively intended as a manual aid for the movability measurement of the upper and the lower extremities.

The goniometer is to be placed on the corresponding body region and is to be adjusted. The angle of movement can now be read by means of the scale of degrees. It must be ensured that the centre of rotation of the goniometer corresponds to the centre of the joint of the respective extremity.

Unsuited use/contraindication of the products: Any use other than that described here is not in accordance with the intended use of the unit. The manufacturer is not liable for any resulting damages. The user alone bears the risk.

Further information, maintenance, storage: When used and stored properly, the goniometer will serve you many years. Never bend the goniometer.

Guarantee: When used properly and in accordance with the User's Manual we guarantee this product for two years from the date of purchase. Should you need further information or should your instrument require repair, please contact your dealer.

Safety instructions: No special requirements. It is recommended that the instruments be retreated immediately after their use.

Limitation of reprocessing: Frequent reuse only minorly influences the instruments. The working life of the instrument usually depends on the wear and tear and damage by use. It is recommended that the instruments be repaired immediately after their use.

Instructions

Workstation: Remove dirt accumulation on surfaces with a disposable cloth.

Storage and transport: In order to avoid dirt accumulation and dust, please always store the goniometer in its packaging.

Cleaning preparation: No special requirements.

Manual cleaning: The goniometer can be cleaned with a damp, soft and lint-free cloth.

Automatic cleaning: Automatic cleaning is not possible.

Disinfection: To disinfect the goniometer, a cloth dampened with alcohol can be used. When wiping the goniometer, only slight pressure should be applied.

Maintenance: Defective parts must be removed and/or replaced.

Check and performance test: The smooth running of all movable parts such as jaw blades must be ensured. Check if the scale is readable.

Packaging: Individual packaging: a standard polyethylene bag can be used. The size of the bag must correspond to the size of the instrument so that it closes properly.

Sterilization: Not a surgical instrument, therefore no sterilization required.

Storage: No special requirements.

Additional information: No special requirements.

Common accessories: Further information about these products can be obtained in our Internet shop at: www.kawemed.de.

Contact: Address or telephone number of the dealer or dial +49-7141-68188-0.

Symbol key:

	Heed the User's Manual
	Manufacturer

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KIRCHNER & WILHELM GmbH + Co. KG