- de** Gebrauchsanweisung
- en** User's Manual
- fr** Mode d'emploi
- it** Istruzioni per l'uso
- es** Instrucciones de empleo
- pt** Manual de operação
- ru** Руководство по применению

MASTERMED C

CE
1304

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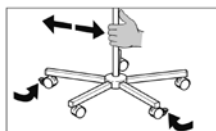
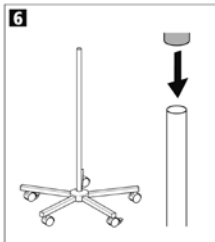
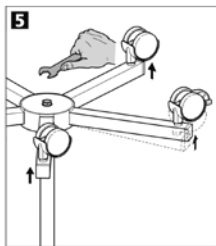
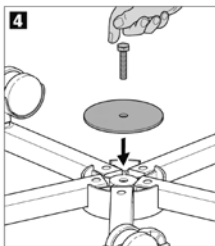
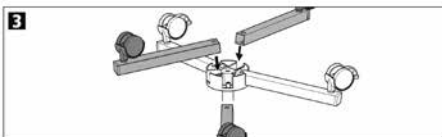
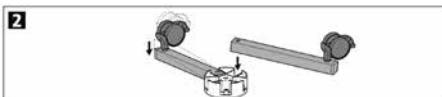
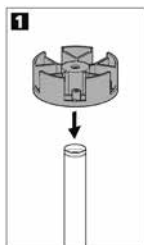
User's Manual: MASTERMED C, Aneroid Blood Pressure Measuring Device (for Doctors)

Dear Customer, thank you for choosing our innovative aneroid blood pressure measuring device. Our products are known for their high quality and longevity. This KaWe product meets the requirements of EC guidelines 93/42/EWG (standards for medical products) and the standard DIN EN ISO 81060-1:2012 "Non-invasive blood pressure measuring devices – Part 1: Requirements and test procedures for non-automated models".

Please read these instructions thoroughly and carefully before attempting to use this product and heed the given care instructions.

Familiarize yourself fully with this product before using it. These instructions are intended for those familiar with the operation of a blood pressure monitor.

Purpose: This device is intended for the non-invasive measurement of systolic and diastolic blood pressure.

Unsuited use/contraindication of the products:

Any use other than that described here is not in accordance with the intended use of the unit. The manufacturer is not liable for any resulting damages. The user alone bears the risk.

Checking the device: Before taking a measurement, insure that the blood pressure monitor is in perfect working order. **1.** Inspect the cuff to ensure that it is not heavily worn or damaged. **2.** Check the instrument gauge dial to make sure that it is within the zero range when the manometer is subject to normal atmospheric pressure. **3.** Ensure that no connection exists between the blood pressure monitor and the intravascular system so that air cannot be accidentally pumped into the blood stream.

Measurement procedure:

1. The patient should sit in a comfortable position with their legs not crossed and their back and arm supported. The operator of the instrument should position him/herself close enough to the patient to be able to place the stethoscope exactly in the correct location and at the same time be able to clearly see the blood pressure monitor display. **2.** Help the patient to relax and remind them not to speak or move during the measurement. The cuff is to be strapped onto the arm such that the middle of the cuff is at the same height as the right atrium. **3.** In order to ensure that you are using the correct cuff size, place the arrow on the cuff over the brachial artery just above the elbow. Close the cuff and check to see if the end of the cuff is within the printed markings. If this is the case then the cuff size is appropriate for the size of the upper arm. A complete assortment of cuff sizes is available. Allow the patient to relax for 5 minutes before the measurement is begun. **4.** Open the air valve in order to insure that the cuff is not under pressure. **5.** Close the air valve. **6.** Estimate the systolic and diastolic blood pressure as follows: **a)** Find the pulse of the brachial artery. **b)** Fill the cuff with air until the pulse disappears. **c)** Make note of the reading. **d)** Release the air from the cuff. **7.** Measure the systolic and diastolic pressure as follows: **a)** Place the stethoscope gently over the spot on the brachial artery where the pulse is the strongest. **b)** Pump up the cuff to approx. 30 mmHg over the previously estimated systolic pressure. **c)** Reduce the pressure by 2 to 3 mmHg per second. Check the actual pressure release rate using the pressure gauge. **d)** Make note of the systolic pressure value as soon as you hear two successive heartbeats. **e)** Make note of the diastolic pressure value as soon as the rhythmic heartbeat sound dissipates (K5).

We recommend K4 for the auscultation of children between the ages of 3 and 12 years old. K4 is when the sound heard with the stethoscope changes from a clear sound to a muffled sound. For pregnant patients, we recommend that K5 be used for auscultation unless the heartbeat can be heard when the cuff is deflated, in which case K4 should be used. Record both measured values rounded up to the next even number (measured in mmHg).

Measure the systolic and diastolic pressure carefully. Repeat the measurement if necessary or if there is ever an uncertainty or distraction. Do not perform the measurement hastily.

Cleaning:

Monitor and accessories (without cuff): To clean the device, first use only a soft dry cloth. The ball, valve and tubes are to be wiped clean with a mild detergent and a damp cloth.

Cuff: If necessary, wipe the cuff clean with a damp cloth.

Disinfection:

Cuff: The cuff can be wiped clean with a damp cloth and soap or disinfectant. The cuffs have been tested for durability with the following recommended disinfecting agents: Buraton (liquid), Cidex, Ethanol 70%, isopropyl alcohol 70%, Mikrozid and Sporidicin. After being disinfected, the cuff is to be rinsed with clean water and air-dried.

Warnings:

1. When the cuff is not connected to the gauge, always use a tube stopper to prevent accidental penetration of liquid. The penetration of liquid into the tube may cause damage to the gauge.
2. Periodically inspect and replace damaged or deteriorated cuffs.

3. Use only the appropriate size cuffs. The arterial pointer must be in the marked area, otherwise the measurement readings may be incorrect.
4. Do not immerse the device in liquid to clean it.
5. Do not sterilize the device!
6. The blood pressure measuring device must be handled carefully and must not be exposed to pressure, sharp objects or any other mechanical stress.
7. The pointer on the gauge must point within the tolerance range of the zero position when there is no pressure in the cuff.
8. Always let all of the pressure out of the cuff before storing the instrument.
9. Protect the surface of the stethoscope diaphragm from damage.
10. Never inflate to more than 300 mmHg.
11. When replacing or repairing parts, please ensure that only original KaWe replacement parts are used.
12. Make sure that the tube system is airtight.

No changes to the device are allowed!

Technical data:

Display/measurement range: 0-300 mmHg

Measurement accuracy: ± 3 mmHg

Monitor:

Storage conditions: -30 °C to +70 °C

Operational conditions: +10 °C to +40 °C

Rel. humidity: 15% - 85%

Cuff:

Storage conditions: -20 °C to +55 °C

Operational conditions: +10 °C to +40 °C

Rel. humidity: 15% - 90%

Altitude: -70 m to +1700m

Arm circumference:

Infant:	13.8 - 21.5 cm
Children:	20.5 - 28.5 cm
Adult:	27.5 - 36.5 cm
Large adult:	36 - 46 cm

Maintenance: It is recommended that the device be handed over to the manufacturer or an authorized service center at least every 2 years for proper metrological control according to the Medical Devices Operator Ordinance. The device must show the cuff pressure with an accuracy of ± 3 mm Hg for the entire measurement range.

Guarantee: When used under normal circumstances and with attention to our user's manual, we guarantee this product for two years after the date of purchase. This product has an expected service life of at least 10 years when used properly and with attention to our manual (instructions).

Accessories:Reusable 2-hose cuff

04.85204.211 - Infants
04.85204.221 - Children
04.85204.231 - Adults
04.85204.241 - Plus-size

Contacting the manufacturer: Address or telephone number of your dealer or dial +49-7141-68188-0.

Further information about this product can be found on our homepage at: www.kawemed.com.









Disposal: The product is to be disposed of in a manner that is compliant with local ordinances.

Symbol key:

	Product reference number
	Lot code
	Batch No
	Date of manufacture
	Please follow the instructions for use
	Manufacturer
	Compliance with relevant guidelines
	Caution!
	Latex free
	Relative humidity
	Temperature range limits

Assembly Instructions for the MASTERMED C Stand model

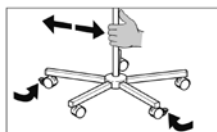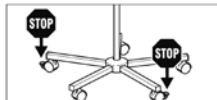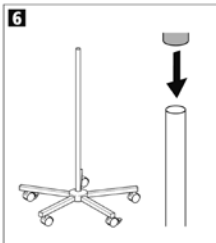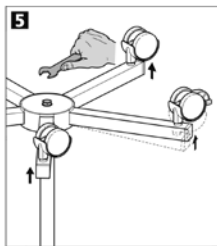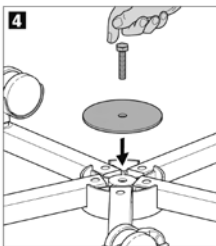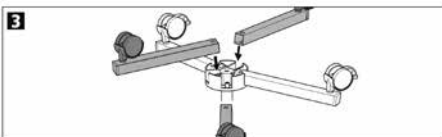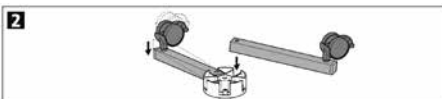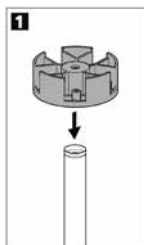1. Attach the block.
2. Insert the two feet with locking castors opposite one another.
3. Insert the three feet with non-locking castors.
4. Screw in hexagon screw.
5. Tighten hexagon screw.
6. Place the MASTERMED C onto the stand post. Lock the castors.
To relocate the stand, release the locking castors.

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KIRCHNER & WILHELM GmbH + Co. KG