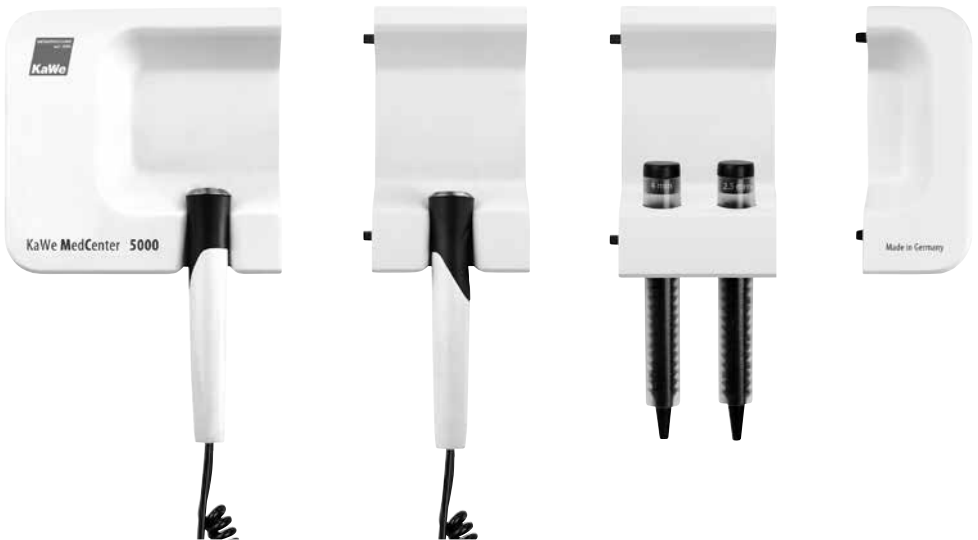
- de** Gebrauchsanweisung
- en** User's Manual
- fr** Mode d'emploi
- it** Istruzioni per l'uso
- es** Instrucciones de empleo
- pt** Manual de operação
- ru** Руководство по применению







Wandstation KaWe MedCenter 5000














QM-1-195G








de	Wandstation KaWe MedCenter 5000	10
en	Wall-Mounted Charging Station KaWe MedCenter 5000	16
fr	Station murale KaWe MedCenter 5000	22
it	Stazione da parete KaWe MedCenter 5000	28
es	Estación de pared KaWe MedCenter 5000	34
pt	Estação de parede KaWe MedCenter 5000	40
ru	Медицинский центр KaWe МедЦентр 5000	46

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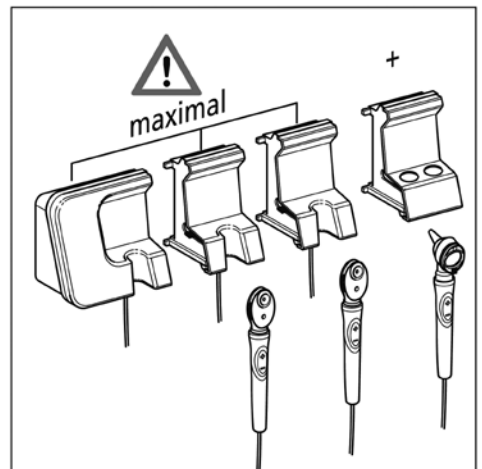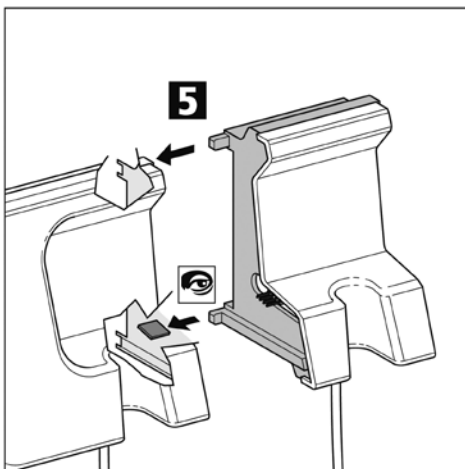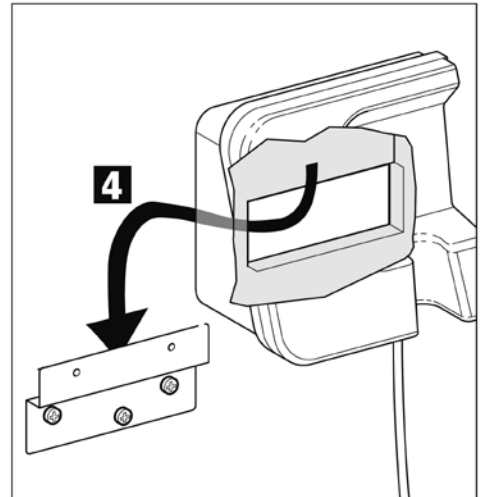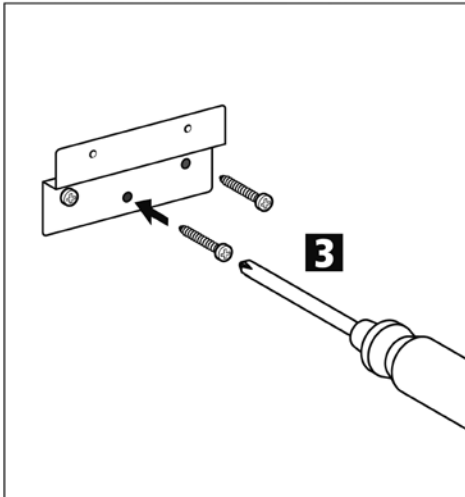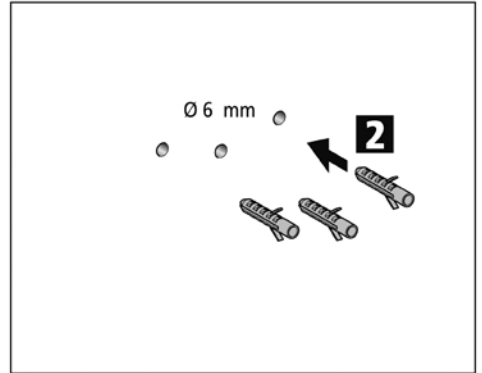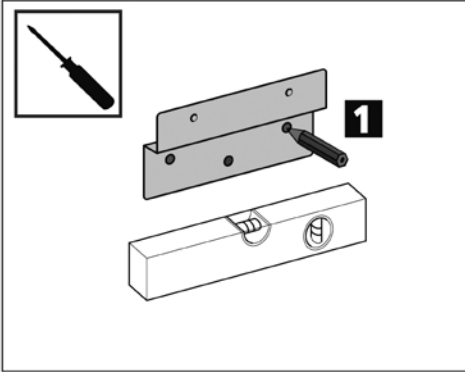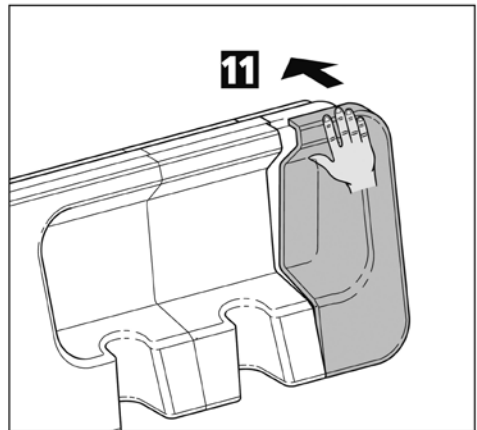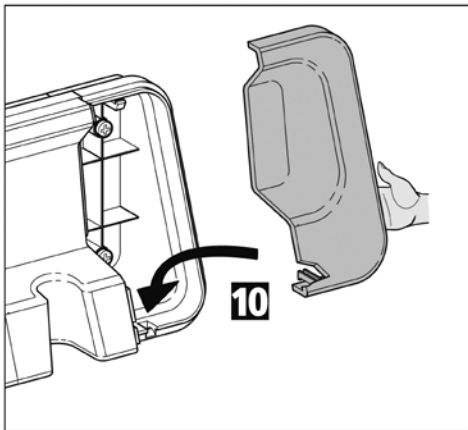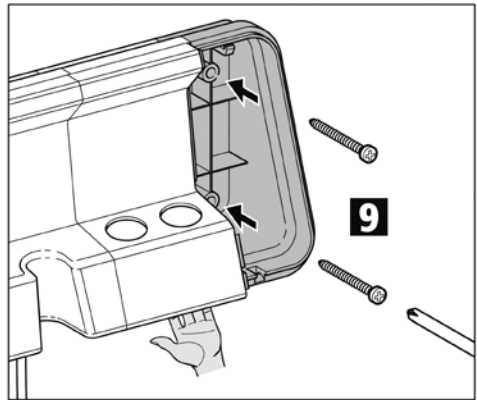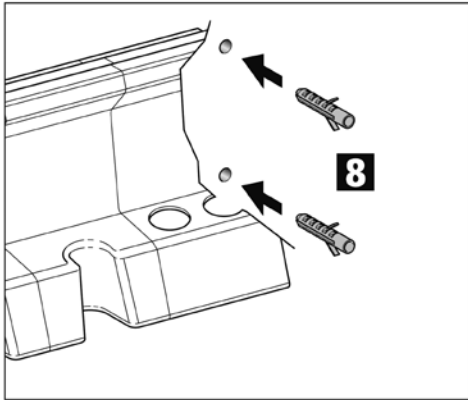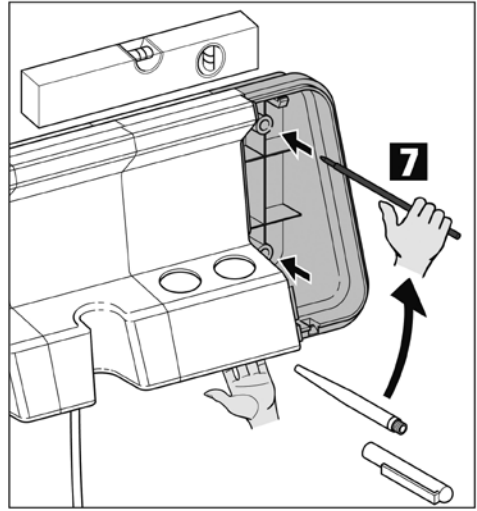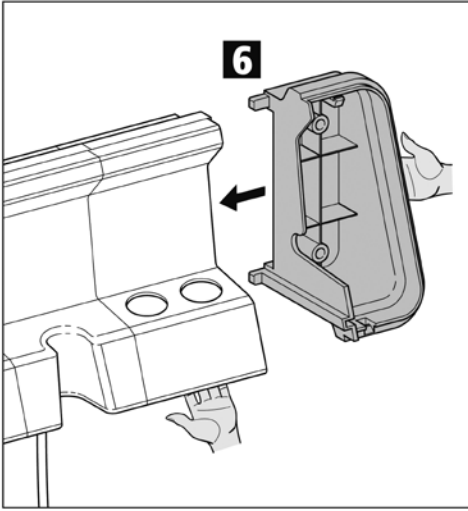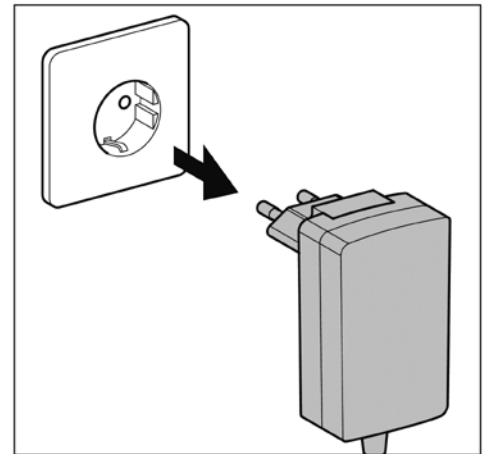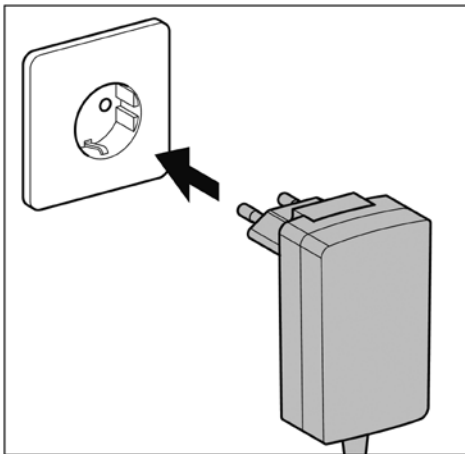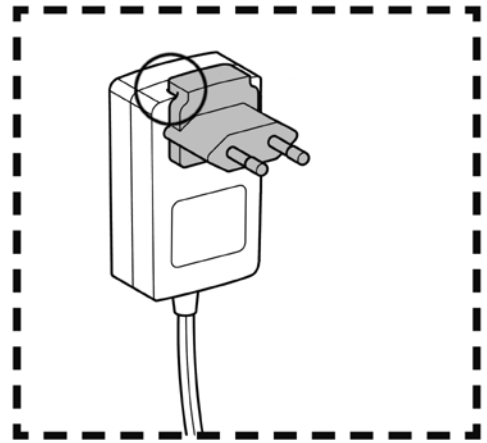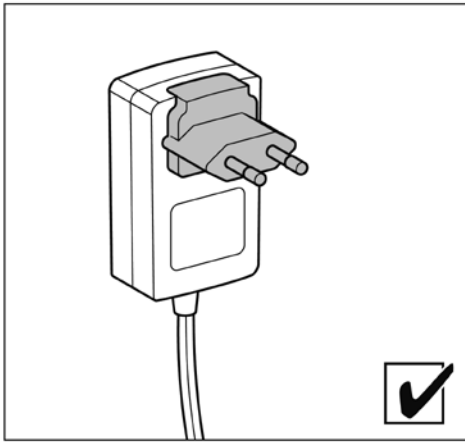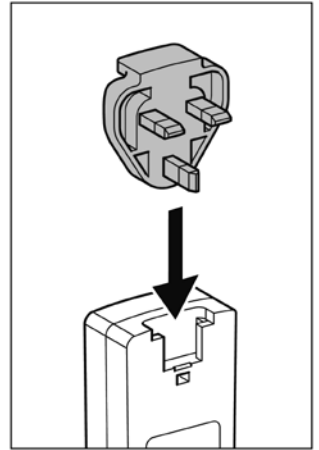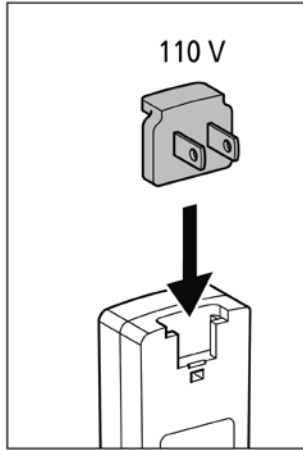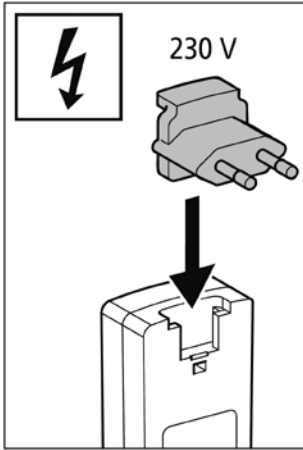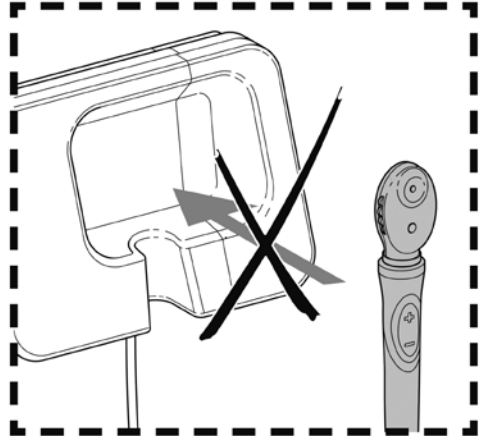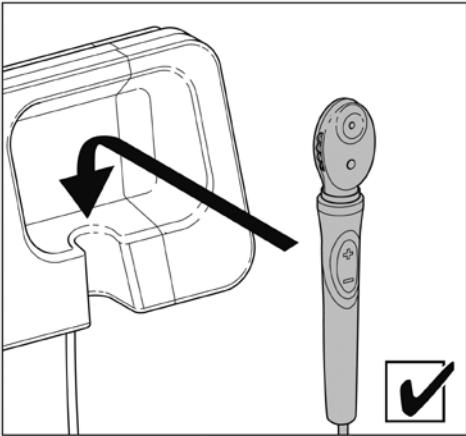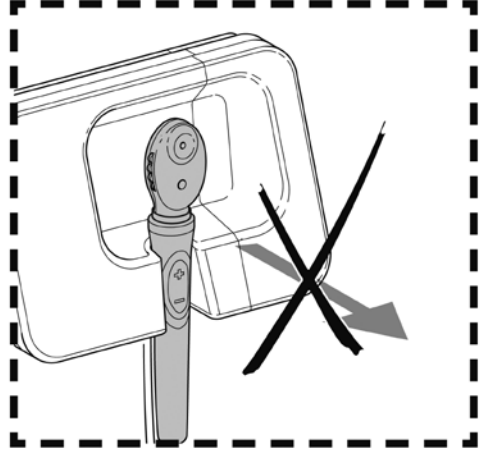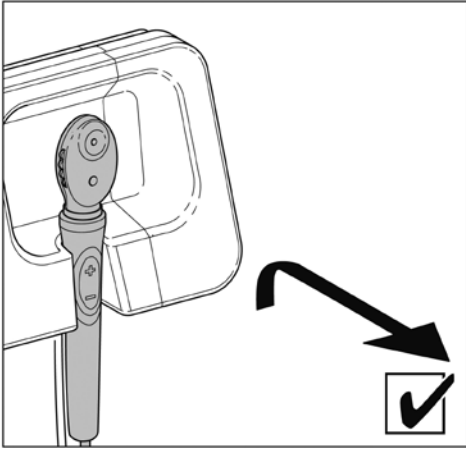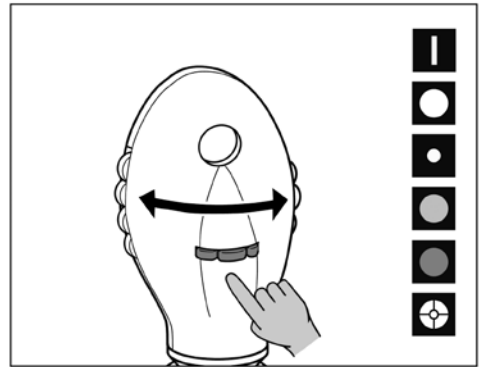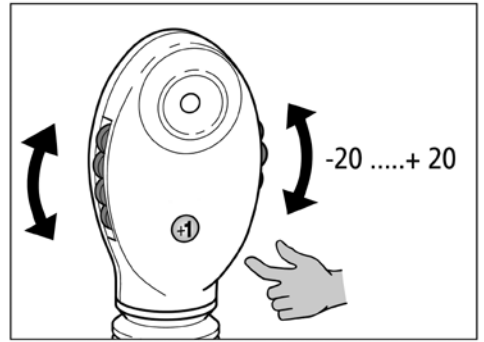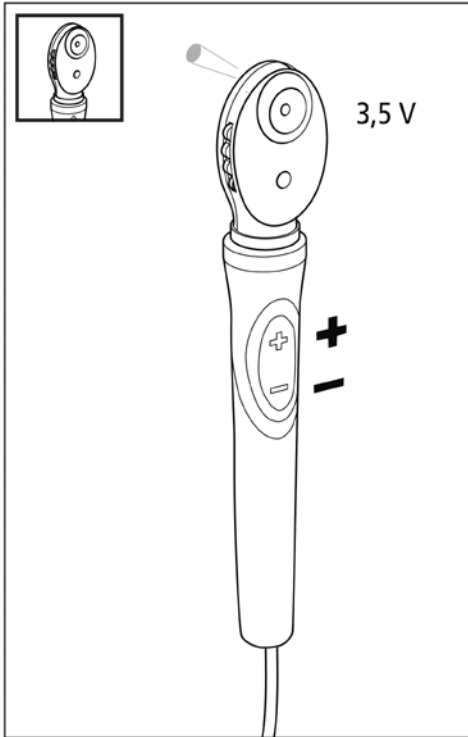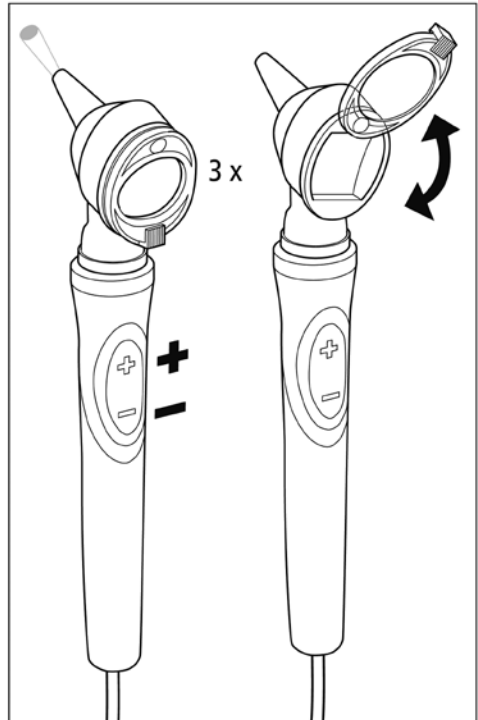
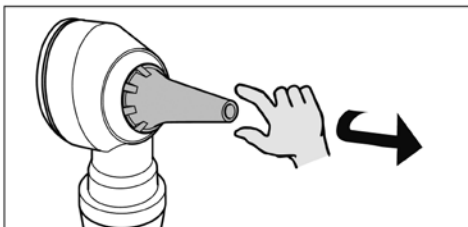
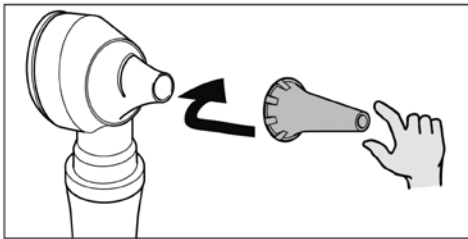
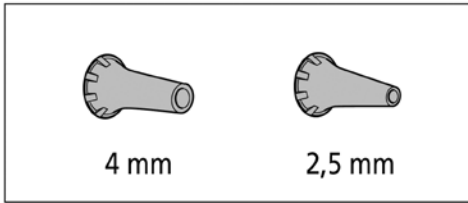
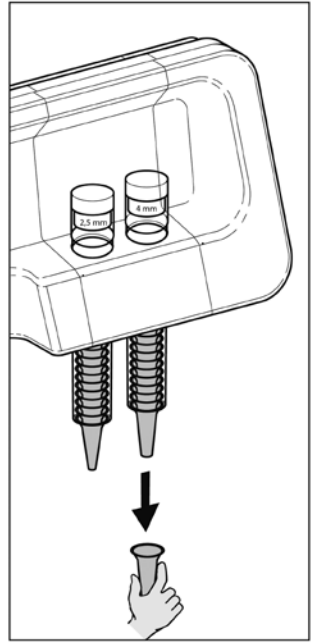
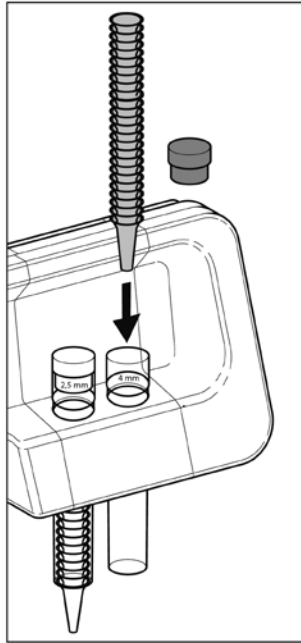
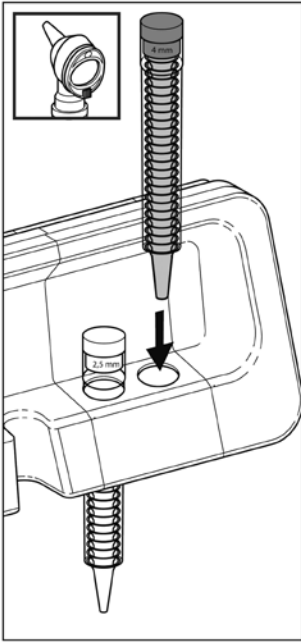
User's Manual Wall-Mounted Charging Station KaWe MedCenter 5000

Dear customer, thank you for choosing a KaWe product. Our products are characterised by their high quality and longevity. This KaWe product complies with the regulations prescribed by EG-Guideline 93/42/EWG (Guideline for Medical Products).

Before using this product, please read this User's Manual carefully and in its entirety and heed the specified main-tenance instructions.

Utilization: This wall-mounted charging station is only to be used by authorized, qualified personnel.

Purpose: This charging station is used to charge diagnostic instruments such as otoscopes and ophthalmoscopes with 3.5 V lamps. The station can be expanded to hold a maximum of three handles.

Assembly/Disassembly: Only the included components are to be used for the assembly of the KaWe MedCenter 5000.

1. Hold the mounting plate level against the wall at the desired mounting location and mark spots for 3 screw anchor holes.
2. With a 6 mm Ø ca. 35 mm drill bit, drill deep holes and insert the anchors.
3. Use a screwdriver and screws to secure the mounting plate to the wall.
4. Place the base unit – disconnected from electric supply – onto the metal lip and while applying pressure, slide it downward gently as far as it will go.
5. Check to be sure that the base unit is securely in its proper position.
6. When attaching an extension module: affix it horizontally on the right side of the base unit. The extension module is secured in place by the guide bar on the base module as well as the frame and its mountings. Hold the extension module and attach its frame to the main frame body and mark the place on the wall for the boreholes. Remove the extension module and frame.
7. With a 6 mm Ø ca. 35 mm drill bit, drill deep holes and insert screw anchors.
8. Mount the expansion module and frame to the base unit as described in no. 6 and secure it with screws.
9. Use a screwdriver and screws to affix the rear panel of the frame to the wall.
10. Attach the front panel of the frame.
11. To disassemble, disconnect the charging station from the power supply. Next, the front panel must be removed. To do this, use a

flat-head screwdriver to gently press the snap-in tab on the bottom of the frame upward and remove the front panel. Remove the screws while at the same time holding on to the modules. Slide the base module upward to remove it from the mounting plate.

Note: The examination head pieces are equipped with a KaWe PICCOLIGHT threaded coupling. These can be screwed onto the handles. To protect against theft, the head pieces are also secured with a side locking screw. To remove the examination head pieces, loosen the locking screw and unscrew the head pieces from the handles.

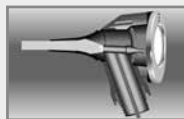
Unintended Use/Contraindication of the Products: Any usage that is different from or that goes beyond the stated intended purpose is considered improper. The manufacturer shall not be liable for damage resulting from improper use; the user alone bears the risk. Only the included handle is to be used to power the instrument.

Initial Use / Handling:

1. Plug the electric cord into the local electric outlet.
2. The green stand-by display on the base module will illuminate.
3. The light in the examination head pieces turns on automatically when the instrument is removed from the station.
4. The brightness can be regulated as necessary with the push button.
5. Two diagnostic instruments can be used at the same time independently of one another. Take care not to cross the power cords.


Replacing the lamps: The proper operation of these diagnostic devices can only be guaranteed when original KaWe lamps are used; they ensure the user excellent light quality!


For the **KaWe PICCOLIGHT C** – otoscope head, slide the magnifying lens up and out of the way and then pull the lamp out by its adapter. Remove the lamp from the adapter and if necessary, clean the glass bulb of the new lamp with alcohol. When inserting the new lamp, be sure to push it in all the way.

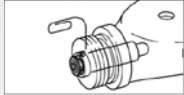Loosen the safety screw on the upper side of the handle. Remove the **KaWe PICCOLIGHT F.O.** – otoscope head from the handle and pull out the lamp. If necessary, clean the glass bulb of the new lamp with alcohol. The bulb should be clean and free of finger prints and oil or grease. The new lamp must be inserted all the way.

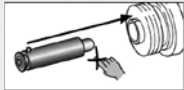Loosen the safety screw on the upper side of the handle. Unscrew the **KaWe PICCOLIGHT** – ophthalmoscope head from the handle. Remove the used lamp and reinsert a new one, ensuring that it is pushed in all the way.

Technical Specifications for the KaWe MedCenter 5000:

Power plug-AC adapter	Type UES12LCP-055200SPA
Wide range input voltage.....	100-240V~, 50/60 Hz
Output voltage.....	5.5 V DC
Output current	max. 2 A
Cord length	3 m
Protection class.....	II

Base module (with frame):

Dimensions of the base module.....	204.5 x 96.5 x 135.5 mm
Spiral power cord length.....	2800 mm (700 mm coiled length)
Weight	ca. 810 g

Extension module:

Dimensions of the additional module.....	80 x 96.5 x 135.5 mm
Spiral power cord length.....	2800 mm (700 mm coiled length)
Weight	ca. 370 g

Ear funnel module:

Dimensions of the ear funnel module.....	80 x 96.5 x 135.5 mm
Weight	ca. 200 g

Environmental condition limits during use:

Temperature	+10 °C to +40 °C
Relative humidity	10% to 75%
Air pressure	700 hPa to 1060 hPa

Storage and transport condition limits:

Temperature	-10 °C to +60 °C
Relative humidity	10% to 95%
Air pressure	700 hPa to 1060 hPa

Warning: This device is maintenance free. For this reason, the housing can only be opened on the side end of the module's frame for the addition of add-on parts. Repairs are only to be carried out by a qualified expert. Never use more than two handles for each KaWe MedCenter 5000 at the same time! The unit does not have any replacement parts.

If damage is apparent, disconnect the power cord immediately. The adapter serves in this case as the separator and must be disconnected from the mains supply if damage is apparent. The adapter must be easily accessible. The charging station is not designed for use in environments in which explosions may occur, in the presence of strong magnetic fields, or in surgery. The spiral cable can cause people to trip. Do not place it where people may be walking. Only KaWe-brand components are to be used. Modifications to this device are not allowed.

While performing examinations, the user is not to touch the conducting parts such as the electric plug, screw closure on the diagnostic unit, etc.

When the light is left on for longer than 1 minute, the metal on the upper part of the handle becomes very hot and when touched can hurt or cause burns. If the handle left on surfaces that are temperature sensitive, there may be a risk of fire. Ensure that after each use, the handle is returned to its holder and the light is turned off. After using for an extended period of time, allow enough time for the handle to cool.

In order to prevent heat buildup on the top part of the handle, the unit turns itself off automatically after 3 minutes. Please activate by pressing the button on the handle or returning to the wall charging station. KaWe otoscopes/ophthalmoscopes are to be used at room temperatures of not greater than 40° C. KaWe otoscopes/ophthalmoscopes are subject to special safety measures with regard to their electromagnetic compatibility. The operation of the KaWe otoscopes/ophthalmoscopes can be influenced by portable and mobile high-frequency communication devices.

Instructions

Storage and Transport: Please store this device in its packaging prior to assembly in order to avoid contact with impurities or dust.

Cleaning Preparation: No specific requirements.

Manual Cleaning: The external parts of the charging station and the diagnostic equipment can be cleaned with a damp, soft, lint-free cloth. Soapy water or other non-aggressive agents may be used.

Automatic Cleaning: No automated cleaning possible / necessary.

Disinfection: The housing of the unit may be wiped with a surface disinfectant. Always wipe the surface dry.

Sterilisation: No sterilisation possible/necessary.

Inspection and Verification of Proper Operation: Inspect the movable parts (such as the magnifier frame and the plug connection) to ensure that they move easily. Turn the diagnostic device on. All devices: perform a visual inspection for damage and wear. Remove and replace damaged parts.

Packaging: Packed in formed Styrofoam, in a cardboard box.

Additional Information: Ensure that the KaWe MedCenter 5000 Wall-Mounted Charging Station is hung in such a way as to minimize the risk of those passing by to be caught on it. The unit should be hung high enough so as to be out of the reach of children.

Manufacturer: **KaWe**

Contacting the manufacturer: Address or telephone number of your dealer, or dial +49-7141-68188-0.

Further Instructions, Maintenance, Storage: When used and stored properly, this product will perform reliably for many years. KaWe otoscopes/ophthalmoscopes are only to be used with KaWe-brand replacement parts and accessories.

Guarantee: When used properly and in accordance with the User's Manual, this product is guaranteed for two years from the date of

purchase. For answers to further questions or for necessary repairs, please contact your dealer.

Common Accessories: Further information about accessories can be found on our homepage at: www.kawemed.com.

Symbol legend:

	Manufacturer
	Date of manufacture
	Lot code
	Separate collection/disposal of electric and electronic devices
	Dispose of expired batteries separately at an appropriate collection centre
	Heed the User's Manual
	Complies with relevant EU guidelines
	Device safety class: II
	Temperature limit
	Keep dry
	GOST-R certification of exports to Russia
	Only use device in dry rooms
	Non-ionizing radiation

Guidelines and manufacturer declaration – electromagnetic emissions

The KaWe MedCenter 5000 is designed to be used in the types of electromagnetic environments listed below. The customer or user is responsible for ensuring that this device is used in such an environment.

Emissions Measurement	Conformity	Electromagnetic environment – guidelines
Type-CISPR 11 – high-frequency emissions	Group 1	The KaWe MedCenter 5000 uses high-frequency energy solely for its internal operation. Its high-frequency emissions are therefore very low and it is unlikely that any nearby electronic equipment will be adversely effected.
Type-CISPR 11 – high-frequency emissions	Class B	The KaWe MedCenter 5000 is designed for use in all facilities including living spaces that are directly connected to a public low-voltage power supply network that also supplies power to buildings used for domestic purposes.
Harmonic oscillations according to IEC 61000-3-2	Class A	
Voltage fluctuations / flicker emissions according to IEC 61000-3-3	Complies	

Guidelines and manufacturer declaration – electromagnetic interference immunity

The KaWe MedCenter 5000 is designed to be used in the types of electromagnetic environments listed below.
The customer or user is responsible for ensuring that this device is used in such an environment.

Interference immunity test	IEC 60601- test level	Compliance level	Electromagnetic environment – guidelines
ELECTROSTATIC DISCHARGE (ESD) according to IEC 61000-4-2	$\pm 6\text{kV}$ contact discharge $\pm 8\text{kV}$ air discharge	$\pm 6\text{kV}$ contact discharge $\pm 8\text{kV}$ air discharge	The flooring should be wood or concrete or be covered with ceramic tile. If the floor material is a non-conductive, synthetic material, the relative humidity of the air must be at least 30%
Fast transient electrical disturbances/ bursts according to IEC 61000-4-4	$\pm 2\text{kV}$ for power supply lines $\pm 1\text{kV}$ for input and output lines	$\pm 2\text{kV}$ for power supply lines $\pm 1\text{kV}$ for input and output lines	The quality of the mains supply voltage should be the same as that of a typical commercial or hospital environment.
Surges according to IEC 61000-4-5	$\pm 1\text{kV}$ phase-to-phase voltage $\pm 2\text{kV}$ phase-to-ground voltage	$\pm 1\text{kV}$ phase-to-phase voltage $\pm 2\text{kV}$ phase-to-ground voltage	The quality of the mains supply voltage should be the same as that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines according to IEC 61000-4-11	$<5\% U_T$ for 1/2 period ($>95\%$ dip) $40\% U_T$ for 5 periods (60 % dip) $70\% U_T$ for 25 periods (30 % dip) $<5\% U_T$ for 5 s ($>95\%$ dip)	$<5\% U_T$ for 1/2 period ($>95\%$ dip) $40\% U_T$ for 5 periods (60 % dip) $70\% U_T$ for 25 periods (30 % dip) $<5\% U_T$ for 5 s ($>95\%$ dip)	The quality of the mains supply voltage should be the same as that of a typical commercial or hospital environment. If the user requires continued operation during power interruptions, it is recommended that the device be powered from an uninterruptable power supply or a battery.
Magnetic field with a power frequency of (50/60 Hz) according to IEC 61000-4-8	3 A/m	3 A/m	Should interference occur, it may be necessary to place the KaWe MedCenter 5000 further from the source line frequency magnetic fields or to fit it with a magnetic shield: the line frequency magnetic field should be measured at the intended use site to ensure that it is not large enough to cause a disturbance.

Note: U_T is the mains voltage prior to application of the test level.

Guidelines and manufacturer declaration – electromagnetic interference immunity

The KaWe MedCenter 5000 is designed to be used in the types of electromagnetic environments listed below. The customer or user is responsible for ensuring that this device is used in such an environment.

Interference immunity test	IEC 60601-test level	Compliance level	Electromagnetic environment – guidelines
Conducted HF-interference according to IEC 61000-4-6	3 V rms 150 kHz to 80 MHz	3 V rms	<p>Portable and mobile HF communications equipment should be used no closer to the KaWe MedCenter 5000 (including its power cords) than the recommended separation distance. The separation distance is calculated using various equations depending on the transmission frequency.</p> <p>Recommended separation distance: $d = 1.2 \sqrt{P}$</p>
HF-interference according to IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	<p>$d = 1.2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2.3 \sqrt{P}$ 800 MHz to 2.5 GHz</p> <p>“P” is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. “d” is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed transmitters, as determined by an electromagnetic survey of the site (a) are less than the compliance level (b) in each frequency range.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> <div style="text-align: center;">  </div>

Note 1: For 80 MHz and 800 MHz, the larger of the two values applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a: Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted with accuracy in theory. It is recommended that in order to assess the electromagnetic environment caused by fixed HF transmitters, an electromagnetic site survey should be conducted. If the measured field strength at the location at which the KaWe MedCenter 5000 is used exceeds the applicable compliance level stated above, the equipment should be checked at each of its locations of use in order to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the KaWe MedCenter 5000.

b: For frequencies ranging from 150 kHz to 80 MHz, the field strength is less than 3 V/m.

Recommended Separation Distances Between Portable and Mobile HF Communications Equipment and the Wall Station KaWe MedCenter 5000

The KaWe MedCenter 5000 is intended for use in an electromagnetic environments in which radiated HF disturbances are controlled. The customer or the user of the equipment can help prevent electromagnetic interference by ensuring that the minimum distance (shown below) between communications equipment (transmitters) and the KaWe MedCenter 5000 is maintained. The minimum distance is dependent on the maximum output power and the frequency of the communications equipment.

Rated Power Output of Transmitter (W)	Separation Distance According to Transmitter Frequencies (m)		
	150 kHz to 80 MHz $d = 1.2\sqrt{P}$	150 kHz to 80 MHz $d = 1.2\sqrt{P}$	150 kHz to 80 MHz $d = 2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.10	0.38	0.38	0.73
1	1.20	1.20	2.30
10	3.80	3.80	7.30
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance can be determined using the equation applicable to the frequency of the transmitter. P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1: An additional factor of 10/3 is used in calculating the recommended separation distance for transmitters in the frequency bands between 80 MHz and 2.5 GHz in order to decrease the likelihood that mobile/portable communications equipment could cause interference if they are inadvertently brought into patient areas.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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KIRCHNER & WILHELM GmbH + Co. KG