- de** Gebrauchsanweisung
- en** User's Manual
- fr** Mode d'emploi
- it** Istruzioni per l'uso
- es** Instrucciones de empleo
- pt** Manual de operação
- ru** Руководство по применению

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**KaWe**

QM-1-029M

<b>de</b>	Gebrauchsanweisung Allzweck-Verband-Schere, Listerschere, Peanklemme .....	4
<b>en</b>	User's Manual All-purpose bandage scissors, Lister scissors, Pean clamp .....	8
<b>fr</b>	Mode d'emploi Ciseaux universels pour pansements, ciseaux de Lister, pinces clamp de Pean .....	12
<b>it</b>	Istruzioni per l'uso Forbici multiuso per bendaggi, forbici Lister, pinze emostatiche Pean .....	16
<b>es</b>	Instrucciones de empleo Tijeras universales para vendajes y tijeras de Lister, pinzas clamp de Pean .....	20
<b>pt</b>	Manual de operação Tesouras para ligaduras e tesouras de Lister, Pinças clamp de Pean .....	24
<b>ru</b>	Руководство по применению Универсальные ножницы для перевязок, ножницы Lister, зажим Pean .....	28

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Dear Customers, thank you for choosing a KaWe product. Our products are known for their high quality and longevity. This KaWe product fulfils EC Standards 93/42/EWG (standards for medical products).

**Please read this User's Manual thoroughly and carefully before attempting to use this product and heed the given care instructions.**

**Purpose:** These instruments are only to be used by authorised professional personnel. The all-purpose bandage and Lister scissors are intended for cutting bandages and articles of clothing. Pean clamps are used for briefly grasping and holding separated blood vessels in order to still the blood flow or before the intended ligation.

**Use:** The all-purpose bandage and Lister scissors are used in general medical practice. Pean clamps are used in surgical procedures.

**Unsuited Use/Contraindication of the Products:** Any use other than that described here is not in accordance with the intended use of the instrument. The manufacturer is not liable for any resulting damages. The user alone bears the risk.

**Further Information, Maintenance, Storage:** When used and stored properly, these instruments will serve you many years.

**Guarantee:** When used properly and with attention to this User's Manual we guarantee this product for two years after the date of purchase. Should you need further information or should your instrument require repair, please contact your dealer!

### **Safety instructions:**

**Limitation of reuse preparation:** Frequent reuse only affects the instruments in a minor way. The working life of the instrument usually depends on the wear and tear and damage due to use. Ultrasonic cleaning is not allowed! Flash-autoclaving and hot air sterilisation will damage the instruments.

**General information:** Please heed the instructions and User's Manual provided by both the manufacturer of the sterilisation device as well the manufacturer of the cleaning/disinfecting agents.

### **Instructions:**

**Storage and Transport:** The instruments are to be put into a tray filled with a suitable cleaning and or disinfecting agent immediately after being used on a patient. Immersing the instruments in a solution keeps residue from drying onto them (protein fixation). It is recommended that the instruments be re-prepared no longer than one hour after their use. The instruments are to be transported to the place at which they are re-prepared in a closed instrument tray.

**Cleaning preparation:** After using the scissors and clamps, we recommend that they be rinsed with running water or with a mildly alkaline solution in order to prevent various kinds of residue (such as blood) from drying on them. In order to properly steam-sterilise (see Cleaning, Sterilisation) the instruments, it is always necessary that they be thoroughly cleaned beforehand!

**Manual cleaning:** According to the recommendations made by the Robert-Koch-Institute (RKI), the preferred method for the further re-preparation of the instruments is mechanical sterilisation. Manual cleaning is not recommended.

### **Automatic cleaning: Equipment:**

CM 310 (Maquet), cleaning agent (neodisher®FA forte 0.4%/neodisher®Z 0.2%)

G7828 (Miele) cleaning agent (Mucapur®XL 0.4 %/Mucapur®Z 0.15 %)

WD 390 (Belimed), cleaning agent (Mucapur®AF 0.5 %/Mucapur®Z 0.1 %)

**1.** Rinse the instruments with running water immediately before they are mechanically cleaned to insure that no residue from cleaning or disinfecting agents is left on the instruments that could enter the machine. **2.** Place the instruments in a suitable instrument stand. **3.** Place the instrument stand in the machine such that the instruments are directly hit by the spray jet. **4.** Put the cleaning agent into the machine as instructed. **5.** Start the Vario TD programme with thermal disinfection. The thermal disinfection programme takes into account both the A0 value and the applicable national regulations (EN/ISO 15883). **6.** After the programme cycle has been completed, remove the instruments from the machine and dry them (as recommended by the RKI, preferably with pressurised air). When using instrument stands, take special care to dry the areas that are hard to reach. **7.** Inspect the instruments to ensure that they are clean and in sound condition with a suitable magnifying device. (Experience has shown that 8-fold magnification allows for optimal visual inspection.) Should there still be leftover residue on the instruments after they have been mechanically treated, repeat the cleaning and disinfection procedures until no more contaminants can be found. **8.** Caution! If the instruments are only cleaned automatically (without verifiable disinfection), a final thermal disinfection of instruments (packed in suitable stands or mesh trays) is necessary.

**Disinfection:** The instruments can be disinfected with Helipur® (or another similar substance). The concentration and reaction times given by the manufacturer must be followed.

**Maintenance:** Defective parts are to be sorted out and/or replaced. Movable parts, such as joints can be greased with surgical lubricating oil.

**Check and performance test:** The smooth running of all movable parts on the scissors and clamps must be ensured. For all instruments: inspect all parts to check for damage and wear.

**Packaging:** Individual packaging: A standard polyethylene bag can be used. The size of the bag must correspond to the size of the instrument so that the bag's closure is not strained.

**Sterilisation:** The instruments can be properly sterilised using the following sterilisation parameters: Fractionated pre-vacuum (3-times), Sterilisation temperature: min. 132 °C, max. 137 °C, Retention time: min. 3 minutes (full cycle), drying time: 10 min.

**Storage:** No special requirements.

**Additional information:** When sterilising different instruments in the autoclave, the maximum load given by the manufacturer must not be exceeded.

**Common Accessories:** Further information about accessories can be found on our homepage at: [www.kawemed.de](http://www.kawemed.de).

Manufacturer: **KaWe**

**Contacting the manufacturer:** Address or phone number of the dealer or dial the number +49-7141-68188-0.

### Symbol key:

	Manufacturer		Heed the User's Manual
	Date of manufacture		Complies with relevant EU-guidelines
	Lot code		Caution!

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**KIRCHNER & WILHELM** GmbH + Co. KG