- de** Gebrauchsanweisung
- en** User's Manual
- fr** Mode d'emploi
- it** Istruzioni per l'uso
- es** Instrucciones de empleo
- pt** Manual de operação
- ru** Руководство по применению

Reflexhammer

QM-1-023H

BUCK

VARIOFLEX

Nadel
Needle
Piquenerf
Ago
Aguja
Agulha
Игла

Pinsel
Brush
Pinceau
Pennellino
Pincel
Pincel
Кисточка

| | | |
|-----------|--------------------------------|----|
| de | Reflexhammer | 4 |
| en | Reflex hammer..... | 7 |
| fr | Marteau à réflexes | 10 |
| it | Martelletto per riflessi | 13 |
| es | Martillo de reflejo | 16 |
| pt | Martelo de reflexos | 19 |
| ru | Молоток неврологический | 22 |

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Dear Customer, thank you for choosing a KaWe product. Our products are known for their high quality and longevity. This KaWe product meets EC Standards 93/42/EEC (standards for medical products).

Please read this User's Manual thoroughly and carefully before attempting to use this product and heed the given care instructions.

Use: These reflex and percussion hammers are only to be used by authorized, trained personnel.

Intended use of instrument: diagnostic instrument used for percussion (tapping on parts of a patient's body by a physician).

Unsuited use/contraindication of the product: Any use other than that described here is not in accordance with the intended use of the instrument. The manufacturer is not liable for any resulting damages. The user alone bears the risk.

Operation of the reflex and percussion hammers: Using the reflex hammer, tap gently with a flick of the wrist on the relevant tendon. This will cause a reflex contraction of the corresponding muscle. The needle and brush are used to test for the various skin reflexes and skin/surface sensitivity.

Further information, maintenance, storage: When used and stored properly, this product will serve you reliably for many years.

Guarantee: When used properly and in accordance with the User's Manual we guarantee this product for two years from the date of purchase. Should you need further information or should your instrument require repair, please contact your dealer.

Limitation of reuse preparation: Frequent re-preparation has only a minimal effect on this product. The end of the product's useful life is normally due to wear and tear and damage by use. It is recommended that instruments be re-prepared as soon as possible after their use.

Instructions

Storage and transport: no special requirements.

Cleaning preparation: no special requirements.

Manual cleaning: Wipe the reflex hammer clean with a damp cloth or a surface cleaning agent. Please heed the concentrations specified by the manufacturer of the cleaning agent.

Disinfection: Disinfectants and cleaning solutions can be used with appropriate concentrations and in accordance with the manufacturer's instructions. If the concentration is too weak, the solution will be ineffective. If the concentration is too strong, it may cause damage to the instrument. Heed the given contact times! Disinfectant and cleaning solutions are to be replaced regularly as they lose their effectiveness with use.

The hammers themselves may not be sterilised. Disinfection is the first step that is to be carried out when cleaning contaminated objects and materials. The goal here is to reduce the number of microorganisms and facilitate the subsequent cleaning process. It is very important to prevent impurities from drying onto the instrument/material. The disinfection process also serves to protect the personnel while they are handling the instruments, as well as the surrounding environment. This selected disinfection process is to be carried out, if possible, directly at the location in which the instruments were used and as soon as possible after their use.

Sterilisation: The needles can be successfully sterilised as follows: fractionated pre-vacuum (3 times), sterilisation temperature: min. 132 °C, max. 137 °C, retention time: at least 3 min (complete cycle), drying time: 10 min.

Maintenance: remove and replace damaged parts.

Inspection and function test: Ensure that movable parts such as needles and brushes can be moved easily. Visually inspect for damage and wear. Before use, ensure that all threaded connections have been screwed on tightly.

Packaging: individually: a standard polyethylene bag can be used. The bag used must be large enough to accommodate the instrument such that the bag closure is not strained.

Storage: no special requirements.

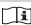
Disposal: This product can be disposed of along with household waste.

Common accessories: Further information about these articles can be found on our homepage: www.kawemed.de.

Manufacturer: **KaWe**

Contacting the manufacturer: Write to or phone your dealer or dial: +49-7141-68188-0.

Symbol key:

| | | | |
|--|------------------------|---|--------------------------------------|
|  | Manufacturer |  | Caution |
|  | Date of manufacture |  | Complies with relevant EU guidelines |
|  | Heed the User's Manual |  | Lot code |

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KIRCHNER & WILHELM GmbH + Co. KG