de	Gebrauchsanweisung	Stethoskope
en	User's Manual	Stethoscopes
fr	Mode d'emploi	Stéthoscopes
it	Istruzioni per l'uso	Stetoscopi
es	Instrucciones de empleo	Estetoscopios
pt	Manual de operação	Estetoscópios
ru	Руководство по применению	Стетоскопы


QM-1-025N

- de** Bitte lesen Sie diese Gebrauchsanweisung vor der Benutzung sorgfältig und vollständig durch und beachten Sie die Pflegehinweise.
- en** Please read this User's Manual thoroughly and carefully before attempting to use this product and heed the given care instructions.
- fr** Lisez ce mode d'emploi attentivement et entièrement avant d'utiliser l'appareil et respectez les consignes d'entretien.
- it** Si prega di leggere attentamente queste istruzioni per l'uso prima di utilizzare lo strumento e di seguire i consigli per la manutenzione.
- es** Por favor, lea con atención las presentes instrucciones de empleo en su totalidad y siga las indicaciones referentes al cuidado del aparato.
- pt** Antes de utilizar este produto pela primeira vez, favor de ler com muita atenção todo este manual de operação e observar as indicações relativas à manutenção.
- ru** Перед использованием следует полностью и внимательно прочитать настоящее руководство по применению и соблюдать указания по уходу!


Deutsch	6-7	
English	8-9	
Français	10-11	
Italiano	12-13	
Español	14-15	
Português	16-17	
Русский	18-19	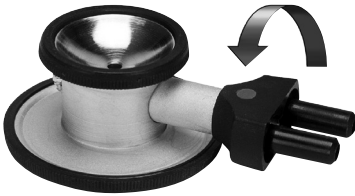

- de** z.B. Noroscop umschaltbar
- en** ex. Noroscop dual-function
- fr** p.ex. Noroscop commutable
- it** ad es. Noroscop con canale selezionabile
- es** p.ej. estetoscopio Noroscop con válvula selectora
- pt** por ex. Noroscop comutável
- ru** например: Noroscop с переключаемой головкой

Lautstärke | volume | intensité sonore | volume |
 intensidad sonora | intensidade sonora | Сила звука

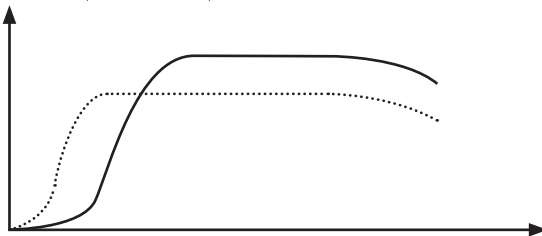Frequenz | frequency | fréquence | frecuencia |
 frecuencia | frequência | Частота

..... Trichter | bell | cloche | campana | campana | campânula | воронку

Membran | diaphragm | membrane | membrana | diafragma | diafragma | мембрану

Profi-Kardiology

Prestige-Standard

Prestige-Standard light

Top-Kardiology

Prestige-Child

Prestige-Child light

Planet

Prestige-Baby

Prestige-Baby light

Planet Air

Kirchner-Colorscope Duo

Double

Rapport/Rapport Black

Kirchner Colorscope Plano

Single

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User's Manual Stethoscopes

Dear Customer, thank you for choosing a KaWe product. Our products are known for their high quality and longevity. This KaWe product fulfils EC Standards 93/42/EEC (standards for medical products).

Please read this User's Manual thoroughly and carefully before attempting to use this product and heed the given care instructions. Familiarize yourself thoroughly with the operation of this unit before using it.

Retain this User's Manual for future use and pass it on to the next user of this device.

Intended Use:

The stethoscopes are only to be used by authorised personnel. The stethoscope consists of binaural pieces, tubes and a chest piece. The stethoscope chest piece is to be placed lightly on the desired area of the patient's body according to the type of examination being performed. Double-sided stethoscopes allow users the option of selecting between two different input channels (bell or diaphragm).

Purpose:

The stethoscope is a diagnostic instrument used for auscultation.

Operation:

The basic principle behind the use of a stethoscope is the "oscillating air column". A closed air column is required for sound to be transmitted. When putting on the binaural stethoscope, adjust the binaural pieces in such a way that the ear tips are pointing slightly forward – toward the patient's face – so that the instrument conforms to the shape of the ear. Place the diaphragm and/or the bell on the area of the body that is to be examined. The following models are equipped with a dual-function diaphragm/bell: Planet Air, Planet, Prestige Stethoskope, Prestige Light Stethoskope, Profi Kardiologie, Kirchner Colorscop

Duo, Double Stethoskop, Rapport Stethoskop and Schwestern-Lehr-Stethoskop Duo. To select between the bell and diaphragm functions, the user need only to twist the head of the stethoscope. In each case, depending on the selected function, one air column is closed and the other is opened.

Stethoscopes with diaphragm heads are used either for auscultation on any uneven surface areas of the body and/or when the auscultation signals in the low frequency range are irrelevant. The bell side of the head is used to diagnose over a wide frequency range. This is mainly required in cardiac auscultation. The edge of the bell must be applied evenly and without any air gaps on all parts of the body it touches.

Unsuitable use/contraindication of the product:

Any use other than that described here is not in accordance with the intended use of the instrument. The manufacturer is not liable for any resulting damages. The user alone bears the risk.

Further information, maintenance, storage:

When used and stored properly, this stethoscope will serve you many years.

Guarantee:

When used properly and in accordance with the User's Manual we guarantee this product for two years from the date of purchase. Should you need further information or should your instrument require repair, please contact your dealer.

Warning instructions:

Never let the stethoscope come into contact with water.

Maintenance and Cleaning:

Workstation: Remove surface impurities with a disposable/paper towel.

Cleaning preparation: No special requirements.

Manual cleaning: The head piece and plastic hoses can be wiped off with alcohol or a mild soap solution. The ear tips are to be unscrewed and can be washed with soapy water. The diaphragm membrane can be washed with soap after the retaining ring has been removed.

Automatic cleaning: Automatic cleaning is not possible.

Disinfection: Disinfecting solutions and cleaning agents such as Meliseptol®/Meliseptol® rapid are to be used in accordance with the manufacturer instructions. Please heed the given reaction times!

Maintenance: Defective parts must be removed and/or replaced.

Check and performance test: Check the diaphragm, tubing and head piece for damage. The instrument must be complete. Before use, the ear tips must be positioned correctly. The position of the head piece is to be adjusted by each user individually.

Packaging: Individual packaging: a standard polyethylene bag can be used. The size of the bag must correspond to the size of the instrument so that it closes properly.

Sterilization: As the stethoscope is not intended for use during operational procedures, regular cleaning procedures are sufficient.

Limitation of reuse preparation:

Frequent reuse only minimally affects the instruments. The working life of the instruments usually depends on the wear and tear and damage by use.

Transport and Storage:

Storage and transport: In order to prevent contamination of the instrument or contact with dust, always store the stethoscope in its case.

Storage: Store the stethoscope in a dry place.

Disposal:

Dispose along with normal domestic waste.

Additional information:

Please always heed these instructions.

Common accessories:

Further information about this product can be found at our internet shop at: www.kawemed.de.

Manufacturer:

Kirchner & Wilhelm GmbH & Co. KG
Eberhardstraße 56 | DE-71679 Asperg

Contact with the manufacturer:

Address or telephone number of the dealer or dial +49 7141 68188-0.

Symbol key:

	Manufacturer
	Date of manufacture
	Lot code
	Heed the User's Manual
	Caution!
	Complies with relevant EU guidelines

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KIRCHNER & WILHELM GmbH + Co. KG