



Vitrex Sterilance Press II

Sterile Safety lancet

Vitrex Sterilance Press II lancet is intended for blood glucose and other in vitro diagnostic tests. Health workers worldwide face accidental needle-stick injury while caring for patients, putting themselves at risk to bloodborne illnesses including the AIDS virus and Hepatitis.

PRESSURE ACTIVATION: Ensures a good blood sample every time.

USER SAFETY: Needle is fully shielded before and after use.


USER CONVENIENCE: Pre-loaded, pressure activated design.

PATIENT COMFORT: High quality, ultra sharp tri-bevel needle or bi-bevel blade, high speed penetration.

EFFICACY: Variety of point configurations and penetration depths to suit most capillary blood requirements.

PATIENT SAFETY: Single-use only. Sterilised by gamma irradiation.



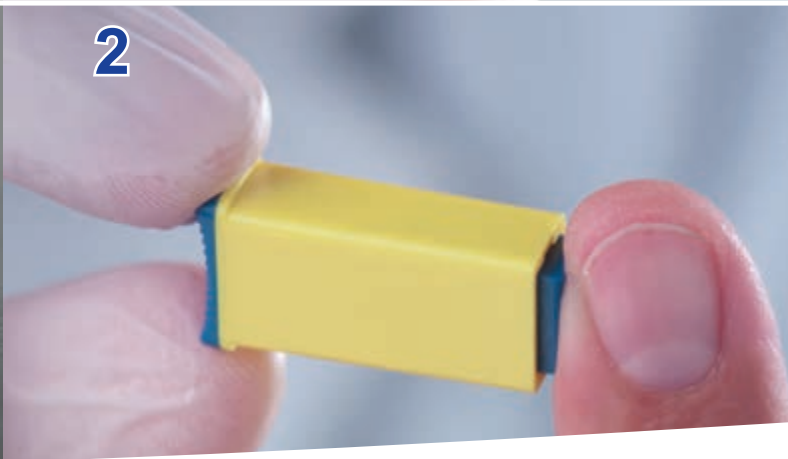

Recommended use

1. Carefully twist off the protective cap.
2. Select puncture site and press the device firmly against the site to release the lancet.
3. Dispose the safety lancet in a suitable safety container.

WARNING: Do not use, if the protective cap has previously been removed.

Technical Specifications

Colour Code	Specification	Penetration Depth	Article No.	Application
Green	18G blade	1.8 mm	393213	Special baby care and maternity units for heel puncture
Blue	21G needle	1.8 mm	397213	As the Yellow lancet, but for increased blood flow
Orange	21G needle	2.2 mm	395213	Haemoglobin (blood banks), cholesterol, blood typing, coagulation, blood gas and other diagnostic tests
Pink	21G needle	2.8 mm	396213	As the Orange lancet, but for thicker adult skin
Yellow	26G needle	1.8 mm	394213	Diabetes clinics/hospital wards for blood glucose monitoring
Purple	28G needle	1.8 mm	398213	Low blood flow, less pain
Turquoise	30G needle	1.8 mm	399213	Low blood flow, less pain

Shelf life 5 years



Packaging:
 Primary packaging: 100 pcs.
 Transit packaging: 1,000 pcs.
 Pallet: 98,000 pcs.