



Cleaning and Disinfection

As influenza and epidemics become more frequent around the world than before, demands to prevent nosocomial infections are becoming important. In particular, Cleaning and disinfecting medical devices are the most fundamental way to prevent infections. We provide these guides to ensure our clients, patients, and clinicians to stay safe and to mitigate infections.

Fetal



ECOtwin LED / ECOtwin LCD / ECOsound / Smart 1 / Smart 3

ECG



Cardio M / Cardio M Plus / PHYSIO PORT
Cardio M-Holter LZ EKG / Cardio E3 / Cardio M-PC

Patient Monitor



COMPACT 5/7/9



PROVIEW 10/12

M10

Cleaning and disinfection should be done in appropriate ways and agents. If medical devices exposed to unapproved agents, it could occur medical-related infections or damage to the device of plastic components. Therefore, we recommend several agents to ensure safe disinfection and cleaning. Please note the devices listed, and find the proper disinfectants as below.

BDM



Excellus / Primus / inus D
inus S

Recommended cleaners and disinfectants for medical ECONET devices

To reduce possibility of device damage and the spread of pathogens, we recommend the following wipe and liquid products. All of these wipes are ready-to-use and alcohol-free products. So they can be used to disinfect the surfaces of alcohol-sensitive medical devices simply and efficiently. And the liquid products are supplied as a concentrate. Therefore, the concentration should be properly adjusted according to the intended use. Also these products can be used with cloths or wipes.

Wipes

MyClean® Wipes DS OA

80.10-5521

Mikrobac® Tissues

80.10-5521

CLEANISEPT® WIPES

80.10-5521

Liquid



HIMID® SinAlc sensitive

46.20-5141

Optisept®

80.10-5521

Mikrobac® forte

80.10-5521

